

**VENOSAN®**

**6000**

Compression Stockings

CLASSIC



**VENOSAN®**


# 6000

Compression Stockings

VENOSAN® 6001  
Cl.I  
18–21 mm Hg  
(2,4–2,8 kPa)

VENOSAN® 6002  
Cl.II  
23–32 mm Hg  
(3,1–4,3 kPa)

The economical stocking which nevertheless satisfies all requirements.

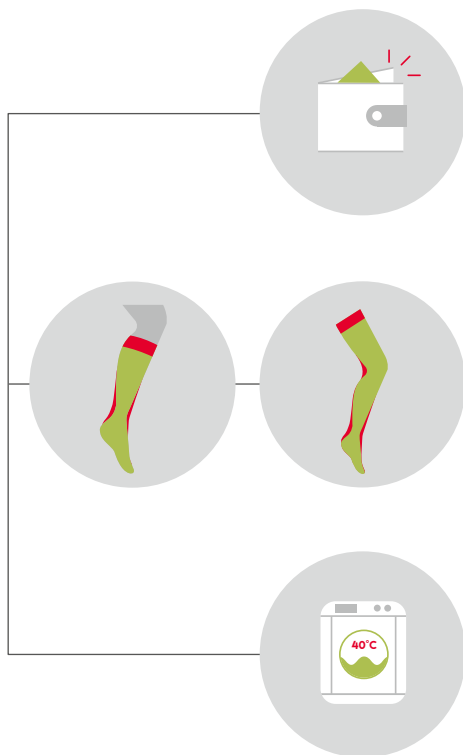

**Simple construction**

Basic yarns make the stocking cost-effective.

**Anatomically knitted**

For maximum comfort, does not slip.

**Ease of care**

Can be machine washed at 40°C on a delicates programme. Dries naturally overnight.

**Colours**

Due to technical printing reasons the colours may vary.






Black




Mexico

70 % Nylon Polyamide  
30 % Elastane



VENOSAN® 6000 Compression Stockings	COMPRESSION CLASS	COLOUR	AD 	AGH 	AT 
	Ccl.I	mexico	▲	▲	▲
black		▲	▲	▲	
Ccl.II	mexico	▲	▲	▲	
	black	▲	▲	▲	

▲ only OTS (off the shelf)

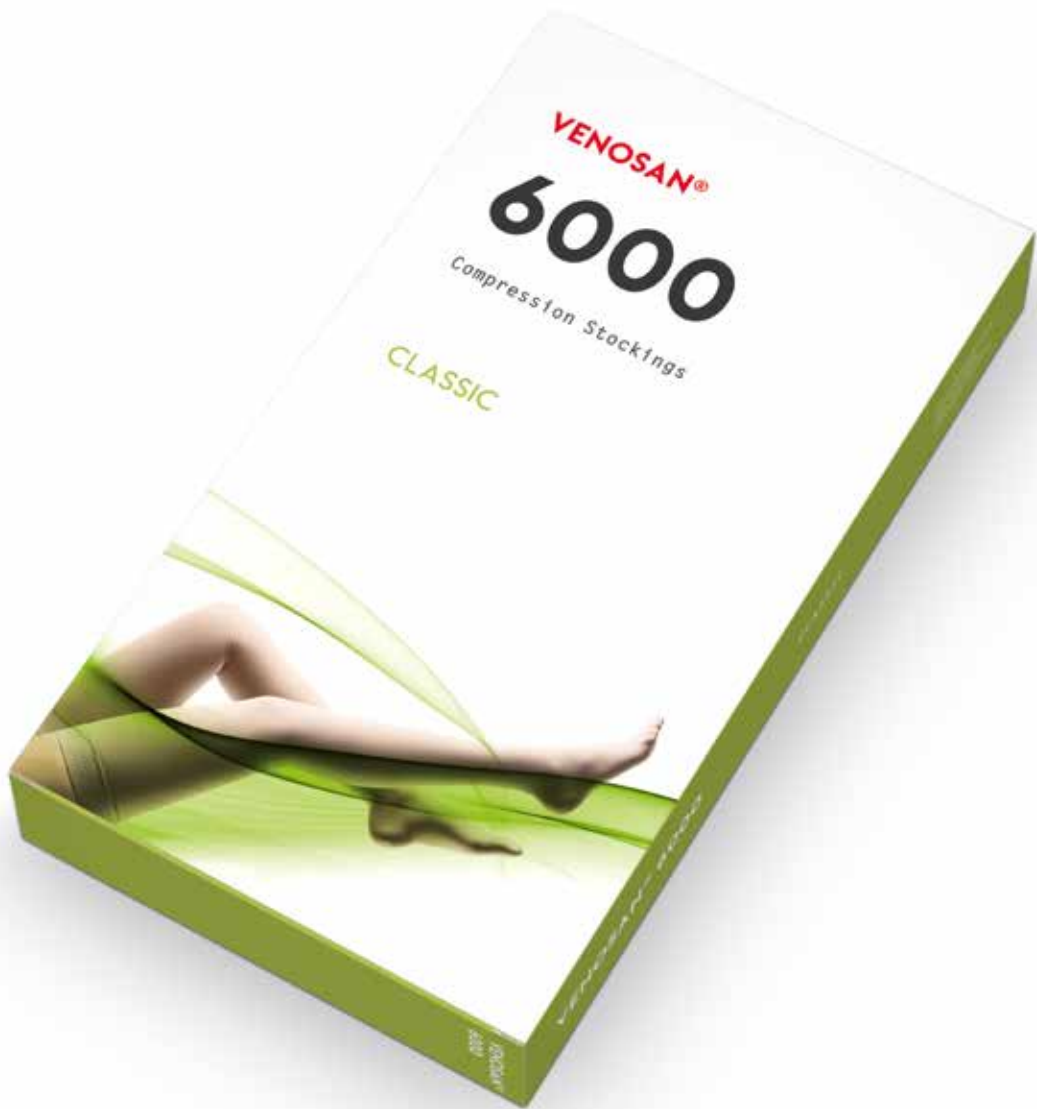

# HOW SHOULD YOU TAKE MEASUREMENTS?

## Compression Stockings

For correct measurements it is very important to take your time and proceed systematically. Whenever possible, the measurements should be taken when the leg is rested and not swollen. The measurements should always be taken with the patient standing. Well fitting medical compression stockings are the key to a successful treatment and satisfied patients.

### MEASURE:



#### Below knee AD

1. the circumference of the ankle cB
2. the circumference of the calf cC
3. the circumference of the leg just below the knee cD
4. the length of the leg from the base of the heel to just below the knee lD



#### Thigh length with self supporting top AGH

1. the circumference of the ankle cB
2. the circumference of the calf cC
3. the circumference of the leg just below the knee cD
4. the circumference of the thigh cG
5. the inside leg lG




#### Thigh length for Pantyhose AT

1. the circumference of the ankle cB
2. the circumference of the calf cC
3. the circumference of the leg just below the knee cD
4. the circumference of the thigh cG
5. the length up to the groin lK



**Measurement Position / Circumference Measurements**

- cG** Thigh (5 cm below groin)
- cD** Narrowest point approx. 2–3 cm below the knee
- cC** Widest part of the calf
- cB** Ankle (narrowest point above ankle bone)

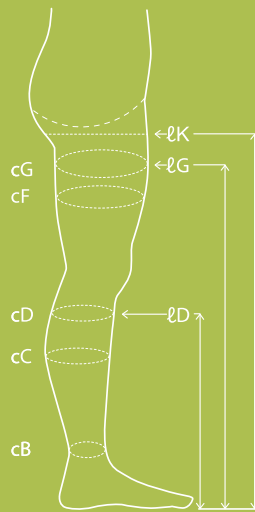

**Length Measurements**

- lK** Up to the groin
- lG** The inside leg
- lD** The length of the leg from the base of the heel to just below the knee

**CIRCUMFERENCE MEASUREMENTS (CM)**

	SMALL	MEDIUM	LARGE	X-LARGE
cB Ankle	20–22	23–25	26–28	29–31
cC Mid-calf	30–39	33–41	36–44	39–47
cD Below the knee	29–38	32–40	35–43	38–46
cG Thigh	49–63	54–66	58–70	63–74

**LENGTH MEASUREMENTS (CM)**

AD	lD	short	34–39
		long	39–44
AGH	lG	short	59–67
		long	68–78
AT	lK	short	67–74
		long	74–85

**SIZE**

It is important that we achieve the correct pressure gradient in accordance with the compression class prescribed by the doctor. The pressure exerted on the leg is determined by 2 factors:

- a) Physical law (LaPlace), which says that with constant tension the pressure is reduced with increased circumference and vice versa.
- b) With increased stretch the tension is increased and vice versa. One effect is therefore partially compensated through the other: Larger circumference = pressure loss due to circumference, but pressure gain due to increased tension of the material.

Thanks to more than 50 years experience we are familiar with these effects and can therefore offer a simple size range in accordance with our sizing chart, which guarantees medical efficacy.

**LENGTH**

VENOSAN® 6000 has a very high stretch in the length and adapts to almost every leg length without slipping. We have also proven through extensive testing that with a difference in length of +/- 3 cm, the pressure changes only by 3–5 %.

**Subsequently compare with the size chart for:**


1. Which size does the ankle circumference cB fall into (small, medium, large or x-large)?
2. Which size do the other measurements fall into cC, cD and cG?

Custom made stockings may not be returned for credit. The manufacturer cannot be held responsible for incorrect measurements.

# 6001

## Compression Stockings Ccl.1

VENOSAN® 6001  
Ccl.1  
18–21 mm Hg  
(2,4–2,8 kPa)

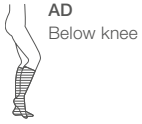

AD  
Below knee



AGH  
Thigh length  
with self  
supporting  
plain top




AT  
Pantyhose

ARTICLE	COLOUR	LENGTH	TOE	SELF SUPPORTING TOP	S	M	L	XL
AD	mexico	long	open		SG 61001	SG 61002	SG 61003	SG 61004
		short			SG 61021	SG 61022	SG 61023	SG 61024
		long	closed		SG 61011	SG 61012	SG 61013	SG 61014
		short			SG 61031	SG 61032	SG 61033	SG 61034
	black	long	open		SN 61001	SN 61002	SN 61003	SN 61004
		short			SN 61021	SN 61022	SN 61023	SN 61024
		long	closed		SN 61011	SN 61012	SN 61013	SN 61014
		short			SN 61031	SN 61032	SN 61033	SN 61034
AGH	mexico	long	open	plain	SG 61601	SG 61602	SG 61603	SG 61604
		short			SG 61621	SG 61622	SG 61623	SG 61624
		long	closed		SG 61611	SG 61612	SG 61613	SG 61614
		short			SG 61631	SG 61632	SG 61633	SG 61634
	black	long	open	plain	SN 61601	SN 61602	SN 61603	SN 61604
		short			SN 61621	SN 61622	SN 61623	SN 61624
		long	closed		SN 61611	SN 61612	SN 61613	SN 61614
		short			SN 61631	SN 61632	SN 61633	SN 61634
AT	mexico	long	closed		SG 61211	SG 61212	SG 61213	SG 61214
		short			SG 61231	SG 61232	SG 61233	SG 61234
	black	long			SN 61211	SN 61212	SN 61213	SN 61214
		short			SN 61231	SN 61232	SN 61233	SN 61234

# 6002

## Compression Stockings Ccl.2

VENOSAN® 6002  
Ccl.II  
23–32 mm Hg  
(3,1–4,3 kPa)

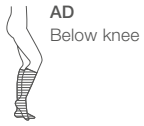

ARTICLE	COLOUR	LENGTH	TOE	SELF SUPPORTING TOP	S	M	L	XL
AD	mexico	long	open		SG 62001	SG 62002	SG 62003	SG 62004
		short			SG 62021	SG 62022	SG 62023	SG 62024
		long	closed		SG 62011	SG 62012	SG 62013	SG 62014
		short			SG 62031	SG 62032	SG 62033	SG 62034
	black	long	open		SN 62001	SN 62002	SN 62003	SN 62004
		short			SN 62021	SN 62022	SN 62023	SN 62024
		long	closed		SN 62011	SN 62012	SN 62013	SN 62014
		short			SN 62031	SN 62032	SN 62033	SN 62034
AGH	mexico	long	open	plain	SG 62601	SG 62602	SG 62603	SG 62604
		short		SG 62621	SG 62622	SG 62623	SG 62624	
		long	closed	plain	SG 62611	SG 62612	SG 62613	SG 62614
		short		SG 62631	SG 62632	SG 62633	SG 62634	
	black	long	open	plain	SN 62601	SN 62602	SN 62603	SN 62604
		short		SN 62621	SN 62622	SN 62623	SN 62624	
		long	closed	plain	SN 62611	SN 62612	SN 62613	SN 62614
		short		SN 62631	SN 62632	SN 62633	SN 62634	
AT	mexico	long	closed		SG 62211	SG 62212	SG 62213	SG 62214
		short			SG 62231	SG 62232	SG 62233	SG 62234
	long	SN 62211			SN 62212	SN 62213	SN 62214	
	short	SN 62231			SN 62232	SN 62233	SN 62234	



**VENOSAN®**  
Compression Stockings

[www.venosan.com](http://www.venosan.com)

VENOSAN® is a registered  
trademark of SWISSLASTIC AG  
ST. GALLEN, Switzerland

an  Company



SWISSLASTIC AG ST. GALLEN  
Rorschacher Strasse 304  
CH-9016 St. Gallen  
Switzerland