

VENOSAN®

8000 ULCERFIT

Compression Stockings System

SUCCESSFUL THERAPY



VENOSAN®

8000 ULCERFIT


VENOSAN®
8000 ULCERFIT
10 mm Hg
(1,3 kPa)

VENOSAN®
8002 ULCERFIT
Ccl.II
23–32 mm Hg*
(3,1–4,3 kPa*)

VENOSAN®
8003 ULCERFIT
Ccl.III
34–46 mm Hg*
(4,5–6,1 kPa*)

Compression Stockings System

Discover how easy it is to treat severe venous disorders with VENOSAN® 8000 ULCERFIT, a two layer medical compression stocking system specifically used to treat and prevent recurrence of venous leg ulcers.

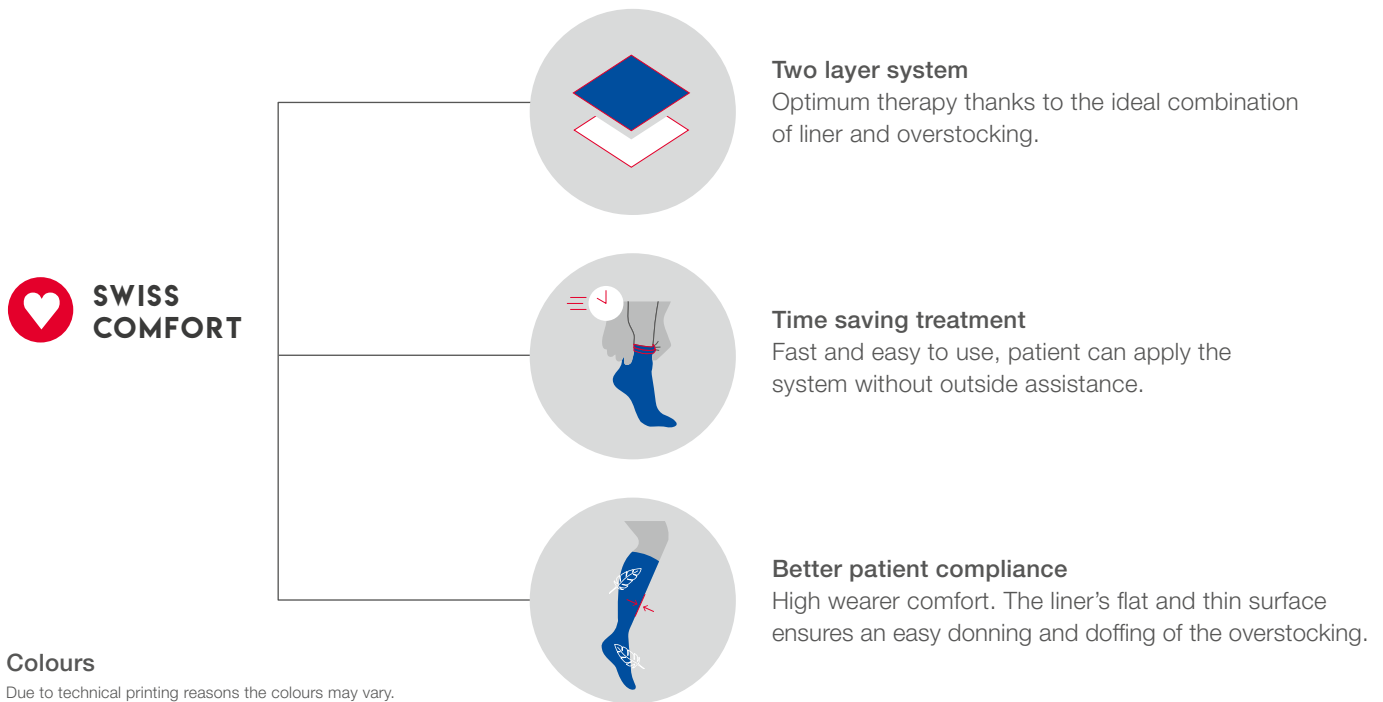


Colours

Due to technical printing reasons the colours may vary.



Beige

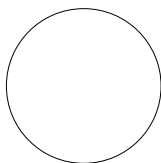

White

Overstocking

62 % TACTEL® Polyamide
26 % Elastane
8 % Cotton/Baumwolle
4 % Lyocell/SeaCell®

Liner

90 % Polyamide
10 % Elastane



VENOSAN® 8000 Ulcerfit Compression Stockings System	COMPRESSION CLASS	COLOUR	AD
	Ccl.II	beige / white	▲
	Ccl.III	beige / white	▲

▲ only OTS (off the shelf)

LINER (APPROX. PRESSURE AT ANKLE 10 MM HG, COLOUR: WHITE)

- Helps to keep the wound dressing in place.
- Smooth surface allows easy donning of the overstocking.
- High wearer comfort due to the low pressure at rest.
- Ideal for wearing during the night.
- Hygienic – washable up to 60°C.
- Available to order separately.



OVERSTOCKING (COLOUR: BEIGE)

- The highly regarded and popular VENOSAN® 7000 compression stocking, also available to wear alone.
- With SeaCell®, the feel good fibre with algae.
- Long-lasting and high stiffness (short stretch).

STOCKING SYSTEM (PRESSURE AT ANKLE 23-32 MM HG IN CCL. II OR 34-46 MM HG IN CCL. III)

- Optimum therapy thanks to the ideal combination of liner and overstocking.
- Simple to wear for the patient.
- Economical.



HOW SHOULD YOU TAKE MEASUREMENTS?

Compression Stockings System

For correct measurements it is very important to take your time and proceed systematically. Whenever possible, the measurements should be taken when the leg is rested and not swollen. The measurements should always be taken with the patient standing. Well fitting medical compression stockings are the key to a successful treatment and satisfied patients.

MEASURE:



Below knee AD


1. the circumference of the ankle cB
2. the circumference of the calf cC
3. the circumference of the leg just below the knee cD
4. the length of the leg from the base of the heel to just below the knee lD



Measurement Position / Circumference Measurements

- cG** Thigh (5 cm below groin) ————— **cG**

- cD** Narrowest point approx. 2–3 cm below the knee ————— **cD**
- cC** Widest part of the calf ————— **cC**
- cB** Ankle (narrowest point above ankle bone) ————— **cB**

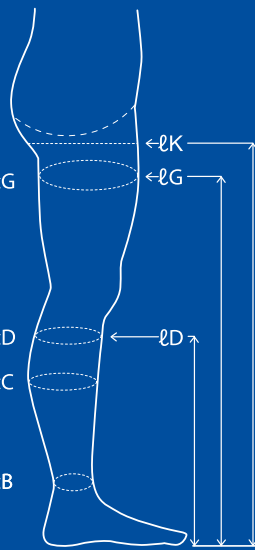

Length Measurements

- lK** Up to the groin
- lG** The inside leg

- lD** The length of the leg from the base of the heel to just below the knee

CIRCUMFERENCE MEASUREMENTS (CM)

	SMALL	MEDIUM	LARGE	X-LARGE
cB Ankle	20–23	23–26	26–29	29–31
cC Mid-calf	29–39	33–42	36–45	39–47
cD Below the knee	28–38	32–41	35–44	38–46

LENGTH MEASUREMENTS (CM)

AD	lD	short	35–40
		long	37–42

SIZE

It is important that we achieve the correct pressure gradient in accordance with the compression class prescribed by the doctor. The pressure exerted on the leg is determined by 2 factors:

- a) Physical law (LaPlace), which says that with constant tension the pressure is reduced with increased circumference and vice versa.
- b) With increased stretch the tension is increased and vice versa. One effect is therefore partially compensated through the other: Larger circumference = pressure loss due to circumference, but pressure gain due to increased tension of the material.

Thanks to more than 50 years experience we are familiar with these effects and can therefore offer a simple size range in accordance with our sizing chart, which guarantees medical efficacy.


LENGTH

VENOSAN® 8000 ULCERFIT has a very high stretch in the length and adapts to almost every leg length without slipping. We have also proven through extensive testing that with a difference in length of +/- 3 cm, the pressure changes only by 3–5%.

8000 ULCERFIT



Liners

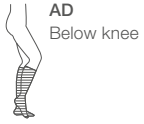

ARTICLE	COLOUR	LENGTH	TOE	S	M	L	XL
AD (Pair)	white	35-42 cm	closed	SG 80001	SG 80002	SG 80003	SG 80004

8002 ULCERFIT

Compression Stockings Set Ccl.2



ARTICLE	COLOUR	LENGTH	TOE (LINERS)	OVER-STOCKING	S	M	L	XL
AD (Set*)	beige/white	long	closed	open	SG 82001	SG 82002	SG 82003	SG 82004
		short	closed	open	SG 82021	SG 82022	SG 82023	SG 82024

* Set containing one (beige) overstocking and two (white) liners

8003 ULCERFIT

Compression Stockings Set Ccl.3



ARTICLE	COLOUR	LENGTH	TOE (LINERS)	OVER-STOCKING	S	M	L	XL
AD (Set*)	beige/white	long	closed	open	SG 83001	SG 83002	SG 83003	SG 83004
		short	closed	open	SG 83021	SG 83022	SG 83023	SG 83024

* Set containing one (beige) overstocking and two (white) liners



VENOSAN®
Compression Stockings

www.venosan.com

VENOSAN® is a registered
trademark of SWISSLASTIC AG
ST. GALLEN, Switzerland

an  Company



SWISSLASTIC AG ST. GALLEN
Rorschacher Strasse 304
CH-9016 St. Gallen
Switzerland