

Compression Wear

Catalogue 2021-22



VENOSAN® Medical Grade Compression Stockings and Garments

VENOSAN® Classes, Styles and Materials	2
VENOSAN® Measurement Size Guide	3
VENOSAN® Selection Chart	4
VENOSAN® Made to Measure Stockings Fax Order Form	6
VENOSAN® Made to Measure	8
VENOSAN® Made to Measure Armsleeves	9
VENOSAN® 4000	10
VENOSAN® 4002 Armsleeves	14
VENOSAN® 5000	16
VENOSAN® 6000	20
VENOSAN® 7000	24
VENOSAN® 7002 Armsleeves	28
VENOSAN® 8000 Ulcerfit	30

VENOSAN® Support Compression Stockings and Socks

VENOSAN® MicroFiberline	34
VENOSAN® Silverline	38
VENOSAN® Legline 20	41
VENOSAN® Legline 30	44
VENOSAN® Athletic	47
VENOSAN® Sportsline	49
VENOSAN® AES	52

Lohmann & Rauscher Lymphoedema Compression Stockings, Ready Wraps and Garments

L&R ReadyWrap™	55
VENOSAN® Flat	64
L&R ExoStrong™	76
L&R ExoSoft™	81
L&R ExoAnklet™	85
L&R TributeWrap™	86
L&R Caresia™	90
L&R SwellSpots™	95

Accessories

Stocking Applicator	104
VENOSAN® Silicone Dotted Donning Gloves	104
VENOSAN® Coltex Roll-On Body Adhesive	104
Ezy-As Compression Stocking Applicator	105
VENOSAN® Socks-Jet Donning and Doffing Aid	105
Eureka-On Compression Stocking Applicator	105
Mediven Butler Off Compression Stocking Remover	106
Doff N' Donner	106
Easy-Slide Compression Arm Stocking Applicator	106
Sama® Compression Therapy	107

VENOSAN®

Medical Grade Compression Stockings and Garments

Classes, Styles & Materials

Classes

COMPRESSION CLASS I	COMPRESSION CLASS II	COMPRESSION CLASS III
MILD COMPRESSION	MODERATE COMPRESSION	FIRM COMPRESSION
18–21 mmHg	23–32 mmHg	34–46 mmHg


Styles



AD
Below
Knee




ADH
Below Knee
with Self
Supporting
Top

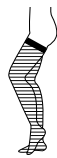

AGH
Thigh High
with Self
Supporting
Plain Top



AGH
Thigh High
with Self
Supporting
Lace Top

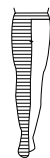

AGG
Thigh Length
with Waist
Attachment



AT
Pantyhose



ATM
Maternity
Pantyhose

Material composition

VENOSAN® 4000: 18–21 mmHg 23–32 mmHg Custom M2M

71% Tactel® Polyamide, 29% Elastane

VENOSAN® 5000: 18–21 mmHg 23–32 mmHg Custom M2M

40% Tactel® Polyamide, 30% Elastane, 20% Cotton, 10% X-Static® – The Silver Fiber™

VENOSAN® 6000: 18–21 mmHg 23–32 mmHg

70% Nylon Polyamide, 30% Elastane

VENOSAN® 7000: 18–21 mmHg 23–32 mmHg Custom M2M (CII, CIII ONLY)

62% Tactel® Polyamide, 26% Elastane, 8% Cotton, 4% SeaCell® Pure


VENOSAN® 8000 Ulcerfit: 23–32 mmHg 34–46 mmHg

OVERSTOCKING: 62% Tactel® Polyamide, 26% Elastane, 8% Cotton, 4% SeaCell® Pure


LINING: 90% Polyamide, 10% Elastane

Measurement Size Guide

CIRCUMFERENCES

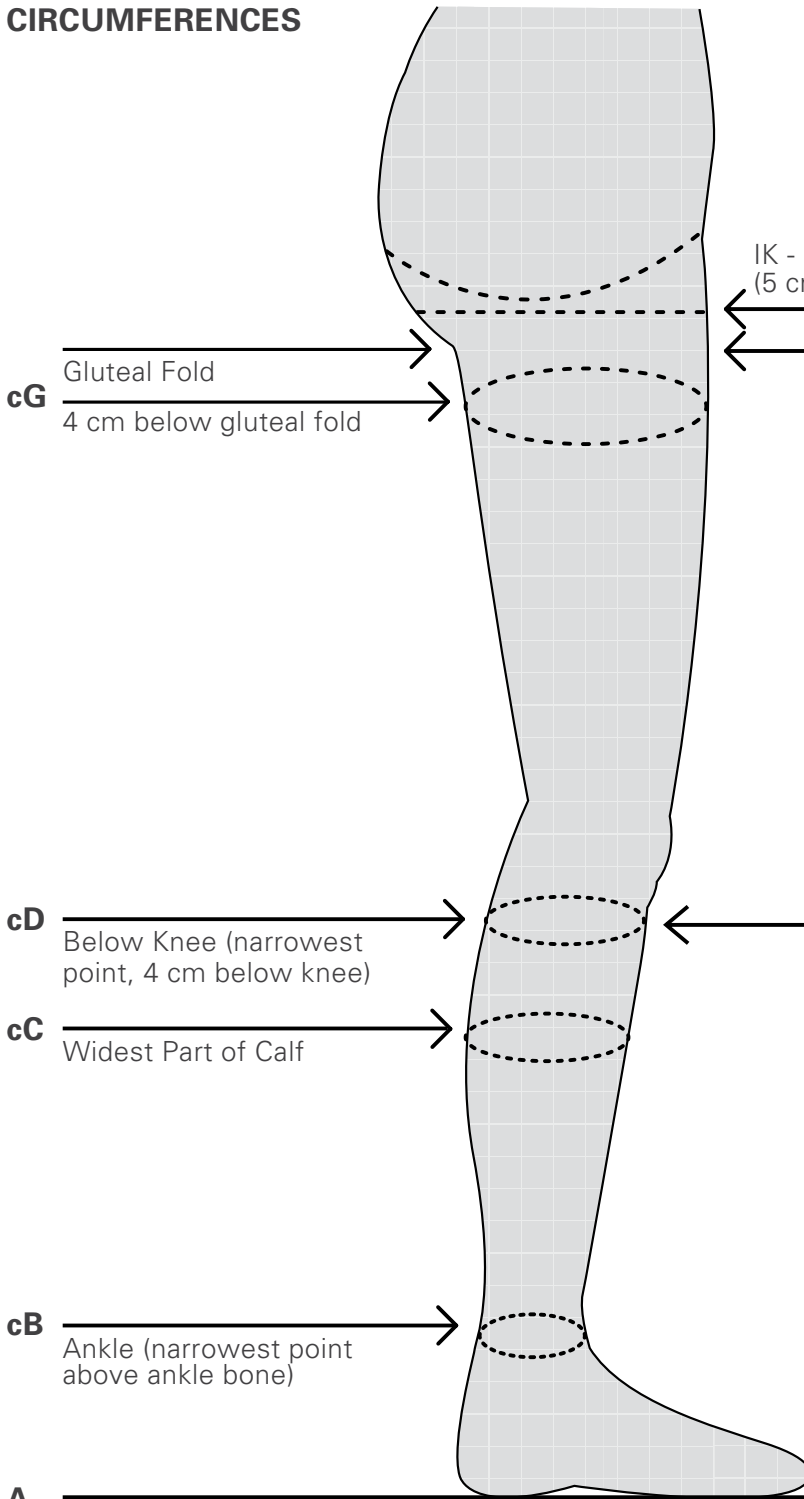

LENGTHS

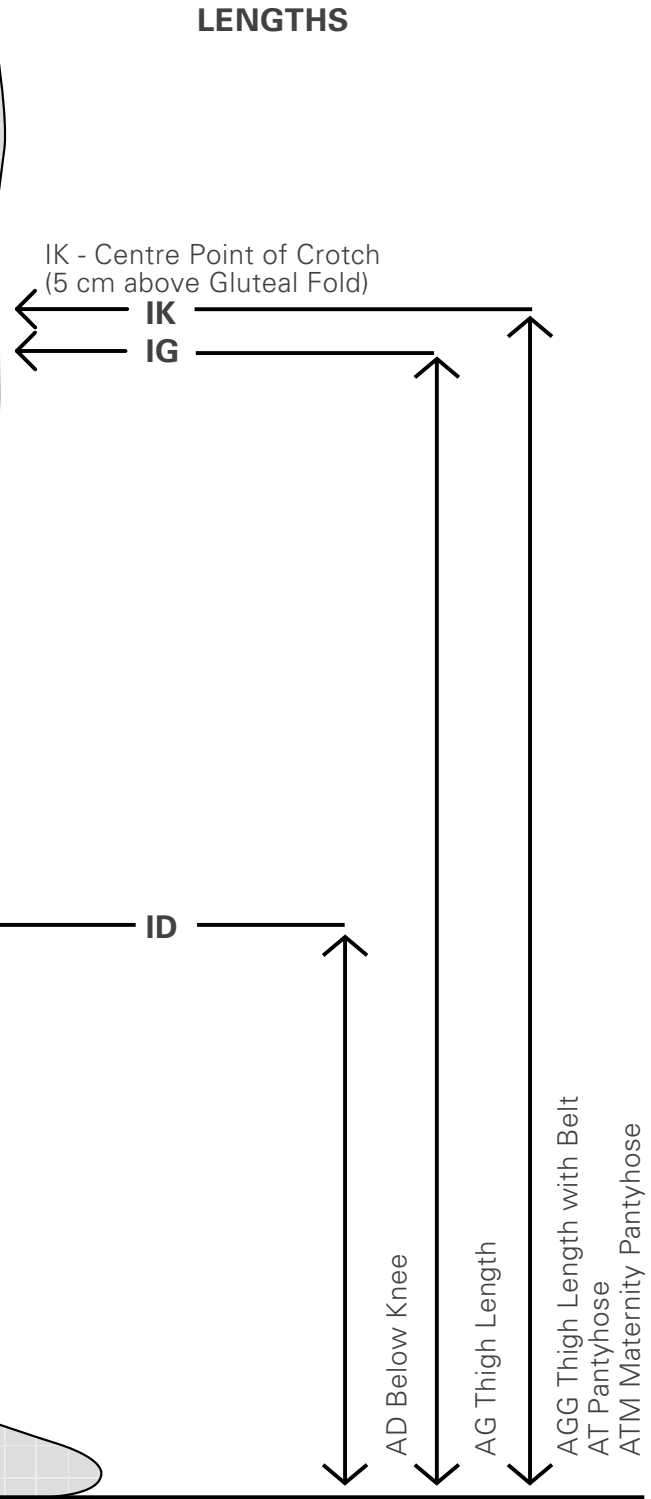

A = Floor

Selection Chart

Orthostatic Stress	CVI	Lipoedema	PTS	Lymphoedema
The term orthostasis literally means standing upright. Orthostatic stress may be defined as the development of symptoms during upright standing relieved by recumbency.	Chronic Venous Insufficiency	Uneven distribution of fat cells in the subcutaneous regions generally in the legs, arms or abdomen	Post Thrombotic Syndrome (long term effects than can occur after venous thrombosis and their consequences)	Localized fluid retention caused by a compromised lymphatic system

Characteristics	Lipoedema	Lymphoedema
Sex	Females only	Males & Females
Age of onset	Puberty	Usually Puberty
Obesity	Yes	Variable
Symmetry	Usually symmetrical	Always symmetrical
Swollen feet	Never	Usually
Skin consistency	Normal or thinner	Thicker & firmer
Pitting of skin	Usually absent	Always pits
Easy bruising	Often	No increase
Pain & Discomfort	Often	Infrequent
Tenderness	Often	Infrequent

Selection Chart

Article	VENOSAN® 7000	VENOSAN® 5000	VENOSAN® 4000
Compression Class	Class I (18–21 mmHg) Class II (23–32 mmHg) Class III (34–46 mmHg) Custom M2M	Class I (18–21 mmHg) Class II (23–32 mmHg) Class III (34–46 mmHg) Custom M2M	Class I (18–21 mmHg) Class II (23–32 mmHg) Custom M2M
Material	62% TACTEL® 26% LYCRA® 8% Cotton 4% SeaCell® Pure	40% TACTEL® 30% LYCRA® 20% Cotton 10% X-STATIC® Silver	71% TACTEL® 29% LYCRA®
Characteristics	<ul style="list-style-type: none"> • Short Stretch • High Stiffness • Antibacterial • Fungicidal • Skin-friendly • SeaCell® Pure is effective against, neurodermatitis • Re-mineralizes the skin • Natural fibres • Open / Closed toe • The feel good fibre 	<ul style="list-style-type: none"> • Long / Medium Stretch • Antibacterial / Antifungal • Silver disperses heat • Enhances microcirculation • Deodorant • Anti-Static • Temperature regulating • Skin friendly • Natural fibres • Open / Closed toe 	<ul style="list-style-type: none"> • Long / Medium Stretch • Climate effect/ wicks away moisture • Breathable / cooler wearing • Fine, almost transparent • Comfortable • Extremely elastic / ease of donning • Skin friendly • Open / Closed toe
Indication	<ul style="list-style-type: none"> • Moderate to severe indications • Secondary varicosities • CVI I, II, III • Lipoedema grade 2-3 • Post-op recovery (Ulceration) • After acute Thrombosis (after bandaging) • PTS (Post Thrombotic Syndrome) • Light lymphoedema • Pronounced orthostatic stress • Inflammatory skin problems 	<ul style="list-style-type: none"> • Mild to Moderate Indications • Primary varicosities • CVI I, II • Sweaty feet • Hot or cold weather • Post-op recovery • Light skin problems (related to veins) • Lipoedema grade 1 	<ul style="list-style-type: none"> • Mild to moderate indications • Primary varicosities • Post sclerotherapy • CVI I • During pregnancy • Light varicose vein symptoms • Post-op
Wearer/Target Group	<ul style="list-style-type: none"> • Men & Women • People whose professions involve intensive periods of standing & demand much from their stockings • Inflammatory skin problems • Long term stocking wearers 	<ul style="list-style-type: none"> • Men & Women • Young & older long term stocking wearers • People that sweat • Clients complaining of excessive heat • People with recurring ulcers • People with athlete's foot or foot odour problems • Clients using walkers that complain of static electricity due to foot dragging 	<ul style="list-style-type: none"> • Men & Women of all ages • Slimmer legs • Arthritic Hands • Clients complaining of excessive heat • Clients that sweat in nylon stockings

Made to Measure Stockings

VENOSAN® 4000

BREATHABLE

Tactel® climate effect wicks away moisture, so ensuring excellent comfort in wear.

EXTREMELY ELASTIC

Easy to put on because of the flat stress/strain curve of the knitted fabric.

COMFORTABLE

Material and visual elegance ensure optimum comfort in wear.

Material (CCL. I, II)

71% Tactel® Polyamide
29% Elastane

Colour

CCL I (Black, Mexico)
CCL II (Black, Mexico, Marokko)

VENOSAN® 5000

COUNTERING ODOUR

X-Static® removes bacteria, so preventing unpleasant odours.

BALANCING FUNCTION

Optimum comfort in wear is achieved by combining X-Static® silver-coated fibre (the high conductivity of silver regulates the temperature) with Tactel® microfibre (rapidly wicks moisture away from the skin).

ANTISTATIC

Because of the high conductivity of silver, X-Static®, keeps you static-free.

Material (CCL. I, II, III)

40% Tactel® Polyamide
30% Elastane
20% Cotton
10% X-Static® – The Silver Fiber™

Colour

CCL I & III (Mexico)
CCL II (Black, Mexico, Marokko)

VENOSAN® 7000

Once again we are able to present a genuine innovation: for the first time SeaCell® Pure fibre is being incorporated in VENOSAN® 7000. This cellulose fibre contains brown algae and has very special effects on human skin.

Benefits

The soft material improves comfort in wear for all those who prefer to feel natural materials against their skin.

- Antibacterial and fungicidal action (inhibits the growth of bacteria and fungi). This effect of SeaCell® Pure is long lasting, even after 150 washing cycles.
- Algae have an anti-inflammatory effect and relieve itching.
- The minerals contained in the algae replenish mineral levels in the skin.

Material (CCL. II, III)

62% Tactel® Polyamide
26% Elastane
8% Cotton
4% SeaCell® Pure

Colour

CCL I & II Black, Beige
CCL III Beige

Made to Measure Stockings and Armsleeves

VENOSAN® 4000 + 7000 MTM

	min.	max.
a	18	50
y	24	52
b	16	50
b1	19	60
c	22	66
d	21	65
e	26	70
f	32	85
g	33	90
h	66	170
t	52	170

VENOSAN® 5000 MTM

	min.	max.
a	18	37
y	24	45
b	16	40
b1	19	50
c	23	56
d	21	55
e	26	62
f	32	82
g	34	88
h	68	150
t	60	150

VENOSAN® MTM ARMSLEEVES

	min.	max.
b	16	40
d	22	59
e	22	
f	22	70
g	23	80

Measuring points

- T** waist
- K** centre point of the crotch
- H** hips (over the buttocks)
- G** thigh (5 cm below K)
- F** mid thigh (between K and E)
- E** mid knee (over the kneecap)
- D** 4 cm below the knee
- C** calf at its maximum girth
- B1** beginning of calf muscle
- B** ankle (narrowest part above the ankle bone)
- Y** heel (foot position 90°)
- A** beginning of small toe

Compression Classes

- I (18–21 mmHg)
- II (23–32 mmHg)
- III (34–46 mmHg)

Measurements

Measurements should be taken with the patient standing and if possible, after the leg has been rested and swelling is at a minimum. All measurements should be made on the bare leg.

Take care

- Custom made stockings may not be returned for credit.
- The manufacturer cannot be held responsible for incorrect measurements.

Guarantee

- We guarantee the premium quality of our VENOSAN® stockings!
- In the event of manufacturing or material faults the stocking will be replaced free of charge within six months.

Made to Measure Stockings Fax Order Form

MEDICAL ACCESSORIES OF AUSTRALIA
11/43 Lang Parade, Milton QLD 4064

P. 1300 527 127
F. 07 3870 5944

sales@medicalaccessories.com.au
www.medicalaccessories.com.au

Name of patient: _____ Mrs Mr Sender (stamp) Customer # _____

Date of birth: _____

Last order no.: _____

Doctor: _____

Customer no.: _____ Date: _____


Quality	Compression class			Style						
	I	II	III	AD	ADH	AG	AGH	AGG	AT	ATM Maternity
VENOSAN® 4000										
VENOSAN® 5000										
VENOSAN® 7000										

Colour: Mexico Beige Black

Special requirements: _____

Quantity: Pair Pieces left Pieces right

Version: plain top w. silicone dots lace top open toe closed toe

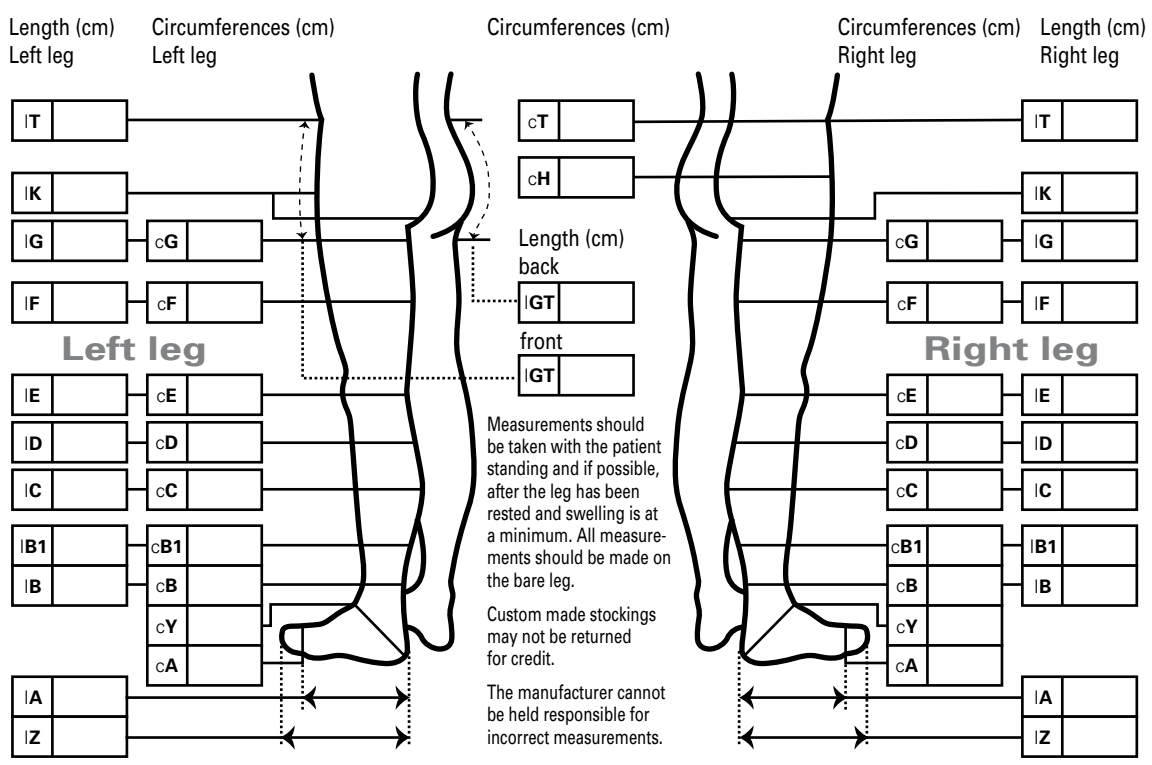

All measurements should be made on the bare leg. The manufacturer is not responsible for incorrect measurements. Custom made stockings cannot be returned.

Made to Measure Armsleeves Fax Order Form

MEDICAL ACCESSORIES OF AUSTRALIA
11/43 Lang Parade, Milton QLD 4064

P. 1300 527 127
F. 07 3870 5944

sales@medicalaccessories.com.au
www.medicalaccessories.com.au

Name of patient: _____ Mrs Mr

Sender (stamp) Customer # _____

Date of birth: _____

Last order no.: _____

Doctor: _____

Customer no.: _____

Date: _____

Quality	Compression class		Style					
	II	III	CG	CG-H	ZG-H	CH	ZG	ZH
				w/ silicone band	w/ silicone band & hand ext.	w/ shoulder strap	w/ hand extension	w/ shoulder strap & hand ext.
VENOSAN® 7000								
VENOSAN® 4000 (Beige only)								

Special requirements _____

Quantity left right | Colour black beige

Length (cm)	Right arm	Circumference (cm)	Left arm	Length (cm)
IH				IH
IG		cG	cG	IG
IF		cF	cF	IF
IE		cE	cE	IE
ID		cD	cD	ID
C		cC	cC	C
IA		cA	cA	IA
IZ		cZ	cZ	IZ

Measure the arm when it is in the relaxed position. The manufacturer is not responsible for incorrect measurements. Custom made armsleeves cannot be exchanged.

VENOSAN®

4000

Compression Stockings and Armsleeves

CLIMATE EFFECT

VENOSAN® 4000 is manufactured in a refined sandwich construction and is the first compression stocking in the world to be awarded the Tactel® climate effect label.

BREATHABLE


Tactel® climate effect wicks away moisture, so ensuring excellent comfort in wear.

EXTREMELY ELASTIC

Easy to put on because of the flat stress/strain curve of the knitted fabric.

COMFORTABLE

Material and visual elegance ensure optimum comfort in wear.

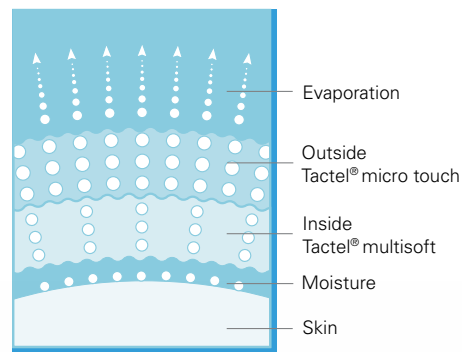

VENOSAN® 4000 Size Chart (cm)

For an accurate fit to your exact measurements, use the chart below

Circumference	Small	Medium	Large	X-Large
cB Ankle	20–23 cm	23–26 cm	26–29 cm	29–31 cm
cC Widest part of calf	29–39 cm	33–42 cm	36–45 cm	39–47 cm
cD 4 cm below the knee	28–38 cm	32–41 cm	35–44 cm	38–46 cm
cG Thigh	48–63 cm	54–67 cm	58–72 cm	63–74 cm
Length				
ID AD/ADH	short	35–40 cm		
	long	37–42 cm		
IG AGH	short	63–72 cm		
	long	68–78 cm		
IK AGG/AT	short	67–74 cm		
	long	74–85 cm		



















VENOSAN® 4001

Compression
CLASS I

Pressure at the ankle
18–21 mmHg

Material: 71% Tactel® Polyamide,
29% Elastane



Colour	Article	Toe	Length		Small	Medium	Large	X-Large
  Black Mexico	AD Below Knee 	Open	long		41001	41002	41003	41004
		Closed	long		41011	41012	41013	41014
		Open	short		41021	41022	41023	41024
		Closed	short		41031	41032	41033	41034
  Black Mexico	ADH Below Knee Silicone top 	Open	long	self supp. plain top	41041	41042	41043	41044
		Closed	long	self supp. plain top	41051	41052	41053	41054
		Open	short	self supp. plain top	41061	41062	41063	41064
		Closed	short	self supp. plain top	41071	41072	41073	41074
  Black Mexico  Marokko *AGH Closed long and Closed short self supporting plain top not available in Mexico.	AGH Thigh High Silicone top  	Closed	long	self supp. lace top	41601	41602	41603	41604
		Open	long	self supp. plain top	41611	41612	41613	41614
		Closed	long	self supp. plain top	41621*	41622*	41623*	41624*
		Closed	short	self supp. lace top	41631	41632	41633	41634
		Open	short	self supp. plain top	41641	41642	41643	41644
		Closed	short	self supp. plain top	41651*	41652*	41653*	41654*
  Black Mexico  Marokko	AT Pantyhose 	Closed	long		41221	41222	41223	41224
		Closed	short		41231	41232	41233	41234
  Black Mexico	ATM Maternity 	Closed	long		41301	41302	41303	41304

VENOSAN® 4002

Compression
CLASS II

Pressure at the ankle
23–32 mmHg

Material: 71% Tactel® Polyamide,
29% Elastane



Colour	Article	Toe	Length		Small	Medium	Large	X-Large
 SG Mexico SN Black	AD Below Knee 	Open	long		42001	42002	42003	42004
		Closed	long		42011	42012	42013	42014
		Open	short		42021	42022	42023	42024
		Closed	short		42031	42032	42033	42034
 SG Mexico SN Black	ADH Below Knee Silicone Top 	Open	long	self supporting plain top	42041	42042	42043	42044
		Closed	long	self supporting plain top	42051	42052	42053	42054
		Open	short	self supporting plain top	42061	42062	42063	42064
		Closed	short	self supporting plain top	42071	42072	42073	42074
 SG Mexico SN Black *AGH Closed long and Closed short self supporting plain top not available in Mexico.	AGH Thigh High Silicone Top 	Closed	long	self supporting lace top	42601	42602	42603	42604
		Open	long	self supporting plain top	42611	42612	42613	42614
		Closed	long	self supporting plain top	42621*	42622*	42623*	42624*
		Closed	short	self supporting lace top	42631	42632	42633	42634
		Open	short	self supporting plain top	42641	42642	42643	42644
		Closed	short	self supporting plain top	42651*	42652*	42653*	42654*
 SG Mexico	AGG LEFT RIGHT 	Open	long		42151	42152	42153	42154
		Open	long		42171	42172	42173	42174
 SN Black SG Mexico SM Marokko	AT Pantyhose 	Closed	long		42221	42222	42223	42224
		Closed	short		42231	42232	42233	42234
 SN Black SG Mexico	ATM Maternity 	Closed	long		42301	42302	42303	42304
		Closed	short		42311	42312	42313	42314

VENOSAN®

4002

Compression Armsleeves

CLIMATE EFFECT

VENOSAN® 4000 is manufactured in a refined sandwich construction and is the first compression stocking in the world to be awarded the Tactel® climate effect label.

BREATHABLE


Tactel® climate effect wicks away moisture, so ensuring excellent comfort in wear.

EXTREMELY ELASTIC

Easy to put on because of the flat stress/strain curve of the knitted fabric.

COMFORTABLE

Material and visual elegance ensure optimum comfort in wear.

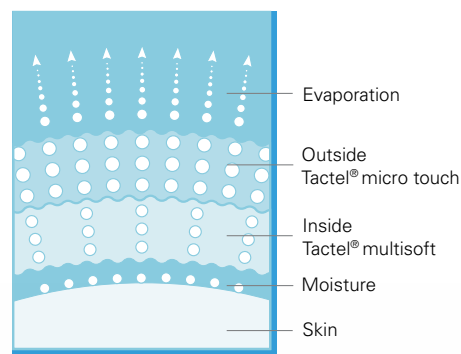

VENOSAN® 4002 Armsleeves Size Chart (cm)

Circumference		Small	Medium	Large	X-Large
cC	Wrist	15–17 cm	17–20 cm	20–23 cm	23–26 cm
cD	Forearm (widest part)	22–26 cm	26–30 cm	30–34 cm	34–38 cm
cG	Upper Arm (Bicep)	24–30 cm	30–36 cm	36–42 cm	42–48 cm

VENOSAN® 4002 Armsleeves

Competition
CLASS II

Pressure at the wrist
23–32 mmHg

Material: 71% Tactel® Polyamide,
29% Elastane



Colour	Article	Length	Hand extension	Self supporting top	Small	Medium	Large	X-Large
 SG Beige	 CG	long	without	without	42501	42502	42503	42504

VENOSAN®

5000

Compression Stockings and Armsleeves

THE MAGIC OF SILVER

VENOSAN® 5000 combines the natural characteristics of cotton with the antibacterial, regulating and antistatic properties of X-Static® and Tactel® high-tech yarns. A top quality product, developed for active women and men.

COUNTERING ODOUR

X-Static® removes bacteria, so preventing unpleasant odours.

BALANCING FUNCTION

Optimum comfort in wear is achieved by combining X-Static® silver-coated fibre (the high conductivity of silver regulates the temperature) with Tactel® microfibre (rapidly wicks moisture away from the skin).

ANTISTATIC

Because of the high conductivity of silver, X-Static®, keeps you static-free.



VENOSAN® 5000 Size Chart (cm)

For an accurate fit to your exact measurements, use the chart below

Circumference	Small	Medium	Large	X-Large
cB Ankle	20–23 cm	23–26 cm	26–29 cm	29–31 cm
cC Widest part of calf	29–39 cm	33–42 cm	36–45 cm	39–47 cm
cD 4 cm below the knee	28–38 cm	32–41 cm	35–44 cm	38–46 cm
cG Thigh	48–63 cm	54–67 cm	58–72 cm	63–74 cm
Length				
ID AD	long	37–42 cm		
IG AGH	long	68–78 cm		
IK AGG/AT	long	74–85 cm		










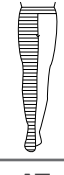




VENOSAN® 5001

Compression
CLASS I

Pressure at the ankle
18–21 mmHg

Material: 40% Tactel® Polyamide, 30% Elastane,
20% Cotton, 10% X-Static® – The Silver Fiber™



Colour	Article	Toe	Length		Small	Medium	Large	X-Large	
 SN Black	 SG Mexico	AD Below Knee 	Open	long		45001	45002	45003	45004
			Closed	long		45011	45012	45013	45014
 SM Marokko									
 SN Black	 SG Mexico	AGH Thigh High Silicone top 	Closed	long	self supp. lace top	45601	45602	45603	45604
			Open	long	self supp. plain top	45611	45612	45613	45614
 SM Marokko									
 SG Mexico		AGG 	LEFT	Open	long	45151	45152	45153	45154
			RIGHT	Open	long	45171	45172	45173	45174
 SN Black	 SG Mexico	AT Pantyhose 	Closed	long		45221	45222	45223	45224
 SM Marokko									

VENOSAN® 5002

Compression
CLASS II

Pressure at the ankle
23–32 mmHg

Material: 40% Tactel® Polyamide, 30% Elastane,
20% Cotton, 10% X-Static®–The Silver Fiber™



Colour	Article	Toe	Length	Small	Medium	Large	X-Large	
<div style="display: flex; justify-content: space-between;"> <div style="text-align: center;"> SG Mexico </div> <div style="text-align: center;"> SM Marokko </div> </div>	AD Below Knee 	Open	long	46001	46002	46003	46004	
		Closed	long	46011	46012	46013	46014	
<div style="text-align: center;"> SN Black </div>								
<div style="display: flex; justify-content: space-between;"> <div style="text-align: center;"> SG Mexico </div> <div style="text-align: center;"> SM Marokko </div> </div>	AGH Thigh High Silicone Top 	Open	long	self supporting plain top	46611	46612	46613	46614
		Closed	long	self supporting plain top	46621	46622	46623	46624
<div style="text-align: center;"> SN Black </div>								
<div style="text-align: center;"> SG Mexico </div>	AGG LEFT RIGHT	Open	long	46151	46152	46153	46154	
		Open	long	46171	46172	46173	46174	
<div style="text-align: center;"> SN Black </div>								
<div style="display: flex; justify-content: space-between;"> <div style="text-align: center;"> SG Mexico </div> <div style="text-align: center;"> SM Marokko </div> </div>	AT Pantyhose 	Closed	long	46221	46222	46223	46224	
		<div style="text-align: center;"> SN Black </div>						

VENOSAN® 5003

Compression
CLASS III

Pressure at the ankle
34–46 mmHg

Material: 40% Tactel® Polyamide, 30% Elastane,
20% Cotton, 10% X-Static®–The Silver Fiber™



Colour	Article	Toe	Length	Small	Medium	Large	X-Large
<div style="display: flex; justify-content: space-between;"> <div style="text-align: center;"> SG Mexico </div> <div style="text-align: center;"> SM Marokko </div> </div>	AD Below Knee 	Open	long	47001	47002	47003	47004
		Closed	long	47011	47012	47013	47014
<div style="text-align: center;"> SN Black </div>							

VENOSAN®

6000

Compression Stockings

CLASSIC

BENEFITS

- Our development team took up the challenge to prove that medical compression stockings do not have to be designed only with the elderly in mind. The result is a stocking which is sheer and elegant, an indispensable companion for the modern person.
- Thanks to its sheerness and the special yarn combination, VENOSAN® 6000 offers a feeling of well-being no matter what the temperature.
- The 4 sizes and 2 lengths, combined with the highly elastic yarns, make for a stocking which sits perfectly on the leg and allows full freedom of movement in all situations.
- VENOSAN® 6000 can be machine washed, like all modern textiles today.

FEATURES

- Breathable
- Sits perfectly
- Complies with medical requirements
- Ease of care



VENOSAN® 6000 Size Chart (cm)

For an accurate fit to your exact measurements, use the chart below

Circumference	Small	Medium	Large	X-Large
cB Ankle	20–22 cm	23–25 cm	26–28 cm	29–31 cm
cC Widest part of calf	30–39 cm	33–41 cm	36–44 cm	39–47 cm
cD 4 cm below the knee	29–38 cm	32–40 cm	35–43 cm	38–46 cm
cG Thigh	49–63 cm	54–66 cm	58–70 cm	63–74 cm
Length				
ID AD/ADH	short	34–39 cm		
	long	39–44 cm		
IG AGH	short	59–67 cm		
	long	68–78 cm		
IK AGG/AT	short	67–74 cm		
	long	74–85 cm		










VENOSAN® 6001

Compression CLASS I

Pressure at the ankle
18–21 mmHg

Material: 70% Nylon Polyamide,
30% Elastane



Colour	Article	Toe	Length		X-Small	Small	Medium	Large	X-Large
  CG Beige CN Black	AD Below Knee 	Open	long		61000	61001	61002	61003	61004
		Closed	long		61010	61011	61012	61013	61014
		Open	short		61020	61021	61022	61023	61024
		Closed	short		61030	61031	61032	61033	61034
  CG Beige CN Black	AGH Thigh High Silicone top 	Open	long	self supp. plain top	61600	61601	61602	61603	61604
		Closed	long	self supp. plain top	61610	61611	61612	61613	61614
		Open	short	self supp. plain top	61620	61621	61622	61623	61624
		Closed	short	self supp. plain top	61630	61631	61632	61633	61634
  CG Beige CN Black	AT Pantyhose 	Closed	long		61200*	61211	61212	61213	61214
		Closed	short		61220*	61231	61232	61233	61234

*AT Closed long and Closed short not available in Black.












VENOSAN® 6002

Compression
CLASS II

Pressure at the ankle
23–32 mmHg

Material:
70% Nylon Polyamide, 30% Elastane

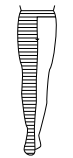

Colour	Article	Toe	Length		X-Small	Small	Medium	Large	X-Large
  CG Beige CN Black	AD Below Knee 	Open	long		62000	62001	62002	62003	62004
		Open	short		62020	62021	62022	62023	62024
		Closed	long		62010	62011	62012	62013	62014
		Closed	short		62030	62031	62032	62033	62034
  CG Beige CN Black	AGH Thigh High Plain Top 	Open	long	self supporting plain top	62600	62601	62602	62603	62604
		Closed	long	self supporting plain top		62611	62612	62613	62614
		Open	short	self supporting plain top	62620	62621	62622	62623	62624
		Closed	short	self supporting plain top		62631	62632	62633	62634
  CG Beige CN Black	AT Pantyhose 	Closed	long			62211	62212	62213	62214
		Closed	short			62231	62232	62233	62234
 CG Beige	AGG 	RIGHT	Open	long		62151	62152	62153	62154
		LEFT	Open	long		62171	62172	62173	62174
		RIGHT	Open	short		62161	62162	62163	62164
		LEFT	Open	short		62181	62182	62183	62184







VENOSAN® 6003

Compression
CLASS III

Pressure at the ankle
34–46 mmHg

Material:
70% Nylon Polyamide, 30% Elastane



Colour	Article	Toe	Length	Small	Medium	Large	X-Large
 CG Beige	AD Below Knee 	Open	long	63001	63002	63003	63004
		Open	short	63021	63022	63023	63024
 CG Beige	AGH Thigh High Plain Top 	Open	long	63601	63602	63603	63604
		Open	short	63621	63622	63623	63624
 CG Beige	AT Pantyhose 	Open	long	63201	63202	63203	63204
		Open	short	63221	63222	63223	63224

VENOSAN®

7000

Compression Stockings and Armsleeves

NATURAL TOUCH

Once again we are able to present a genuine innovation: for the first time SeaCell® Pure fibre is being incorporated in VENOSAN® 7000. This cellulose fibre contains brown algae and has very special effects on human skin.

BENEFITS

- Antibacterial and fungicidal action (inhibits the growth of bacteria and fungi). This effect of SeaCell® Pure is long lasting, even after 150 washing cycles.
- Algae have an anti-inflammatory effect and relieve itching.
- The minerals contained in the algae replenish mineral levels in the skin.
- Short stretch.



VENOSAN® 7000 Size Chart (cm)

For an accurate fit to your exact measurements, use the chart below

Circumference	Small	Medium	Large	X-Large
cB Ankle	20–23 cm	23–26 cm	26–29 cm	29–31 cm
cC Widest part of calf	29–39 cm	33–42 cm	36–45 cm	39–47 cm
cD 4 cm below the knee	28–38 cm	32–41 cm	35–44 cm	38–46 cm
cG Thigh	48–63 cm	54–67 cm	58–72 cm	63–74 cm
Length				
ID AD/ADH	short	35–40 cm		
	long	37–42 cm		
IG AGH	short	63–72 cm		
	long	68–78 cm		
IK AGG/AT	short	67–74 cm		
	long	74–85 cm		



VENOSAN® 7001

Compression
CLASS I

Pressure at the ankle
18–21 mmHg

Material: 62% Tactel Polyamide,
26% Elastane, 8% Cotton, 4% SeaCell® Pure



Colour	Article	Toe	Length	Small	Medium	Large	X-Large
 SG Beige		Open	long	71001	71002	71003	71004
		Closed	long	71011	71012	71013	71014
		Open	short	71021	71022	71023	71024
		Closed	short	71031	71032	71033	71034












VENOSAN® 7002

Compression
CLASS II

Pressure at the ankle
23–32 mmHg

Material: 62% Tactel® Polyamide,
26% Elastane, 8% Cotton, 4% SeaCell® Pure

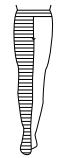

Colour	Article	Toe	Length		Small	Medium	Large	X-Large	
 SG Beige  SN Black	AD Below Knee 	Open	long		72001	72002	72003	72004	
		Closed	long		72011	72012	72013	72014	
		Open	short		72021	72022	72023	72024	
		Closed	short		72031	72032	72033	72034	
 SG Beige  SN Black	ADH Below Knee Silicone Band 	Open	long		72041	72042	72043	72044	
		Closed	long		72051	72052	72053	72054	
		Open	short		72061	72062	72063	72064	
		Closed	short		72071	72072	72073	72074	
 SG Beige  SN Black	AGH Thigh High Silicone Top 	Open	long	self supporting plain top	72601	72602	72603	72604	
		Closed	long	self supporting plain top	72611	72612	72613	72614	
		Open	short	self supporting plain top	72621	72622	72623	72624	
		Closed	short	self supporting plain top	72631	72632	72633	72634	
 SG Beige	AGG 								
			long		72151	72152	72153	72154	
		LEFT	Open	short		72161	72162	72163	72164
			long		72171	72172	72173	72174	
	short		72181	72182	72183	72184			
	RIGHT	Open	long		72171	72172	72173	72174	
	short		72181	72182	72183	72184			


VENOSAN® 7003

Compression
CLASS III

Pressure at the ankle
34–46 mmHg

Material: 62% Tactel® Polyamide,
26% Elastane, 8% Cotton, 4% SeaCell® Pure



Colour	Article	Toe	Length	Small	Medium	Large	X-Large	
  SG Beige SN Black	AD Below Knee	Open	long	73001	73002	73003	73004	
		Closed	long	73011	73012	73013	73014	
		Open	short	73021	73022	73023	73024	
		Closed	short	73031	73032	73033	73034	
		ADH Below Knee Silicone Band	Open	long	73041	73042	73043	73044
			Closed	long	73051	73052	73053	73054
			Open	short	73061	73062	73063	73064
			Closed	short	73071	73072	73073	73074
	AGH Thigh High Silicone Top	Open	long	73601	73602	73603	73604	
		Closed	long	73611	73612	73613	73614	
		Open	short	73621	73622	73623	73624	
		Closed	short	73631	73632	73633	73634	

VENOSAN®

7002


Compression Armsleeves

NATURAL TOUCH

Once again we are able to present a genuine innovation: for the first time SeaCell® Pure fibre is being incorporated in VENOSAN® 7000. This cellulose fibre contains brown algae and has very special effects on human skin.

BENEFITS

- Antibacterial and fungicidal action (inhibits the growth of bacteria and fungi). This effect of SeaCell® Pure is long lasting, even after 150 washing cycles.
- Algae have an anti-inflammatory effect and relieve itching.
- The minerals contained in the algae replenish mineral levels in the skin.
- Short stretch.

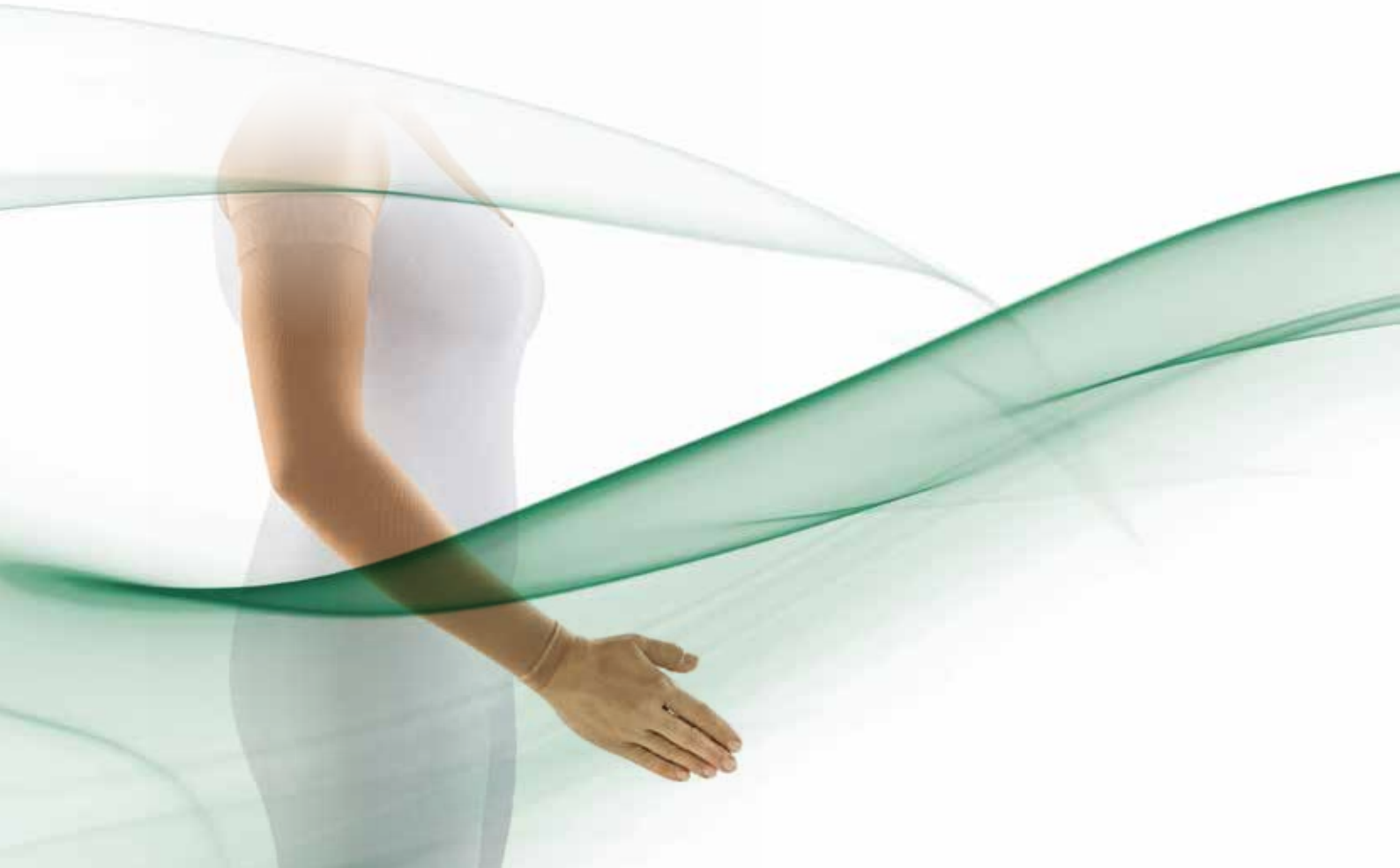

VENOSAN® 7002 Armsleeves Size Chart (cm)

Circumference		Small	Medium	Large	X-Large
cC	Wrist	14–17 cm	17–20 cm	20–23 cm	23–26 cm
cD	Forearm (widest part)	22–26 cm	26–30 cm	30–34 cm	34–38 cm
cG	Upper Arm (Bicep)	24–30 cm	30–36 cm	36–42 cm	42–48 cm
Length					
C to G	Wrist to Upper Arm	short	35–45 cm		
		long	40–50 cm		

VENOSAN® 7002 Armsleeves

Compression
CLASS II

Pressure at the wrist
23–32 mmHg

Material: 62% Tactel® Polyamide,
26% Elastane, 8% Cotton, 4% SeaCell® Pure



Colour	Article	Length	Hand extension	Self supporting top	Small	Medium	Large	X-Large
 SG Beige	 CG	long	without	without	72501	72502	72503	72504
		short	without	without	72511	72512	72513	72514
 SG Beige	 CG-H	long	without	plain	72521	72522	72523	72524
		short	without	plain	72531	72532	72533	72534
 SG Beige	 ZG	Long	with	without	72541	72542	72543	72544
		Short	with	without	72551	72552	72553	72554
 SG Beige	 ZG-H	Long	with	plain	72561	72562	72563	72564
		Short	with	plain	72571	72572	72573	72574

VENOSAN®

8000 ULCERFIT

Compression Stocking System

SUCCESSFUL THERAPY

VENOSAN® 8000 ULCERFIT is a compression stockings system for the treatment of leg ulcers (crural ulcer). The treatment system consists of two specially knitted stockings. They complement each other in their medical effect and generate a special pressure configuration.

BENEFITS

- The liner is normally worn day and night. This stops wound dressings from slipping.
- The overstocking supports treatment during the day by guaranteeing the defined pressure.

When treating a leg ulcer (wound) your doctor integrates the stocking system into the correct phase of wound management, which always includes a suitable dressing.



VENOSAN® 8000 Size Chart (cm)

For an accurate fit to your exact measurements, use the chart below



Circumference	Small	Medium	Large	X-Large
cB Ankle	20–23 cm	23–26 cm	26–29 cm	29–31 cm
cC Widest part of calf	29–39 cm	33–42 cm	36–45 cm	39–47 cm
cD 4 cm below the knee	28–38 cm	32–41 cm	35–44 cm	38–46 cm
Length				
ID AD	short	35–40 cm		
	long	37–42 cm		

VENOSAN® 8002 Ulcerfit

*One overstocking open toe 18–21 mmHg and two white liners closed toe 10 mmHg

Compression CLASS II |
 Pressure at the ankle w/Liner **23–32 mmHg** |
 Material: 62% Tactel® Polyamide, 26% Elastane, 8% Cotton, 4% SeaCell® Pure






Colour	Article	Length		Small	Medium	Large	X-Large
 SG Beige	AD Below Knee 	long 37–42 cm	14 ½ – 16 ½ in.	82001	82002	82003	82004
		short 35–40 cm	13 ¾ – 15 ¾ in.	82021	82022	82023	82024

VENOSAN® 8003 Ulcerfit

*One overstocking open toe (23–32 mmHg) and two white liners closed toe (10 mmHg)

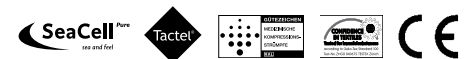Compression CLASS III |
 Pressure at the ankle w/Liner **34–46 mmHg** |
 Material: 62% Tactel® Polyamide, 26% Elastane, 8% Cotton, 4% SeaCell® Pure



Colour	Article	Length		Small	Medium	Large	X-Large
 SG Beige	AD Below Knee 	long 37–42 cm		83001	83002	83003	83004
		short 35–40 cm		83021	83022	83023	83024



VENOSAN® 8000 Liners

Closed toe for liners
Pressure at the ankle 10 mmHg

Compression
10 mmHg

Material: 90% Polyamide,
10% Elastane



Colour	Article	Length	Small	Medium	Large	X-Large
 SG White	AD Below Knee	35–42 cm / 13¾–16½ in.	80001	80002	80003	80004
						

VENOSAN®

Support Compression Stockings and Socks

VENOSAN®

MICROFIBERLINE

Compression Support Socks

STYLISH COMFORT, FASHIONABLY SOFT

VENOSAN® MicroFiberline is a graduated compression support sock with a closed toe. Micro Fiber softness and excellent breathability ensure wearer comfort. For active men and women in work and travel.

- Graduated compression
- Stylish comfortable support
- Fashionably soft comfort
- Reinforced heel and toe
- Micro Fiber softness
- Comfortable non-restricting top band





VENOSAN® MicroFiberline Men

SUPPORT
CLASS

Pressure at the ankle
15–20 mmHg

Material:
82% Nylon, 10% Spandex (Latex Free), 8% Cotton



Swatch	Article	Colour	Toe	Small	Medium	Large	X-Large
 MC Black	AD Below Knee	Black	closed	79002	79003	79004	79005
							

VENOSAN® MicroFiberline Size Chart (cm)

For an accurate fit to your exact measurements, use the chart below.

	Small	Medium	Large	X-Large
Ankle	19–21.5 cm	21.5–24 cm	24–26.5 cm	26.5–32 cm
Calf	28–38 cm	33–43.5 cm	35.5–46 cm	38.5–51.5 cm

VENOSAN® MicroFiberline Shoe Size Chart

For persons with large calf muscles, it is recommended that the next larger size be used.

	Small	Medium	Large	X-Large
US	7 & under	7½–10	10½–12	12½+




VENOSAN® MicroFiberline Lady

SUPPORT
CLASS

Pressure at the ankle
15-20 mmHg

Material:
90% Nylon, 10% Spandex (Latex Free)

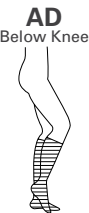

Swatch	Article	Colour	Toe	Small	Medium	Large	X-Large	
 MC Black	 MC Beige	 AD Below Knee	Black	closed	79221	79222	79223	79224
			Beige	closed	79205	79206	79207	79208

VENOSAN® MicroFiberline Size Chart (cm)

For an accurate fit to your exact measurements, use the chart below

	Small	Medium	Large	X-Large
Calf	26.5–37 cm	29–39.5 cm	32–43 cm	34.5–47 cm
Ankle	16.5–20.5 cm	20.5–24.5 cm	24.5–28 cm	28–32 cm

VENOSAN® MicroFiberline Shoe Size Chart

For persons with large calf muscles, it is recommended that the next larger size be used

	Small	Medium	Large	X-Large
US	4–5	5½–7½	8–10½	11–13



VENOSAN® MicroFiberline Men

SUPPORT
CLASS

Pressure at the ankle
20–30 mmHg

Material:
82% Nylon, 10% Spandex (Latex Free), 8% Cotton



Swatch	Article	Colour	Toe	Small	Medium	Large	X-Large
 MC Black	AD Below Knee 	Black	closed	89002	89003	89004	89005

VENOSAN® MicroFiberline Men Size Chart (cm)

For an accurate fit to your exact measurements, use the chart below

	Small	Medium	Large	X-Large
Calf	28–38 cm	33–43.5 cm	35.5–46 cm	38.5–51.5 cm
Ankle	19–21.5 cm	21.5–24 cm	24–26.5 cm	26.5–32 cm

VENOSAN® MicroFiberline Men Shoe Size Chart

For persons with large calf muscles, it is recommended that the next larger size be used

	Small	Medium	Large	X-Large
US	7 & under	7½–10	10½–12	12½+

VENOSAN®

SILVERLINE


Compression Support Socks

EVERYDAY COMFORT

SILVERLINE® offers long-lasting protection from foot odour and athlete's foot. X-Static®, the silver fibre, promotes hygiene and anti-microbial properties. Skin remains deodorized and free of bacteria and fungus—particularly effective in protecting against foot odour and athlete's foot. Cool and dry in Summer and warm in Winter, SILVERLINE® fibres wick moisture away from the skin for comfort in warm weather. X-Static®, the silver fibre, conducts heat in Winter to retain warmth. The silver in X-Static® yarns is an excellent conductor and eliminates static electricity.

SPECIFICALLY DESIGNED

- Enhanced comfort and fit with knitted top band
- Elimination of bacteria and fungus associated with foot odour
- Excellent regulator of heat

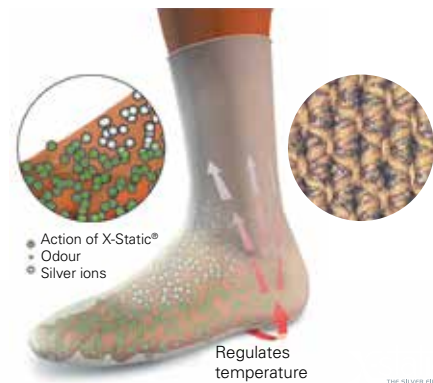



VENOSAN® Silverline Gentleman

SUPPORT
CLASS

Pressure at the ankle
20–30 mmHg

Material: 86% Nylon, 9% Spandex, 5% X-STATIC®
Silver Plated Nylon (Latex Free)

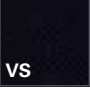

Swatch	Article	Colour	Toe	Small	Medium	Large	X-Large	XX-Large
 vs Black	AD Below Knee	Black	closed	49405	49406	49407	49408	49409
								

VENOSAN® Silverline Gentlemen Size Chart (cm)

For an accurate fit to your exact measurements, use the chart below

	Small	Medium	Large	X-Large	XX-Large
Calf	28–38 cm	33–43.5 cm	35.5–46 cm	38.5–51.5 cm	41–53 cm
Ankle	19–21.5 cm	21.5–24 cm	23–26.5 cm	26.5–32 cm	32–36 cm

VENOSAN® Silverline Gentlemen Shoe Size Chart

For persons with large calf muscles, it is recommended that the next larger size be used

	Small	Medium	Large	X-Large	XX-Large
US	7 & under	7½–10	10½–12	12½–14	13–15




VENOSAN® Silverline Lady

SUPPORT
CLASS

Pressure at the ankle
20–30 mmHg

Material: 80% Nylon, 11% Spandex,
9% X-STATIC® Silver Plated Nylon (Latex Free)

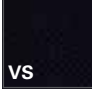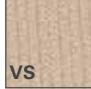

Swatch	Article	Colour	Toe	Small	Medium	Large	X-Large	XX-Large
 VS Black  VS Beige	AD Below Knee 	Black	closed	49505	49506	49507	49508	49509
		Beige	closed	49511	49512	49513	49514	49515

VENOSAN® Silverline Lady Size Chart (cm)

For an accurate fit to your exact measurements, use the chart below

	Small	Medium	Large	X-Large	XX-Large
Ankle	16.5–20.5 cm	20.5–24.5 cm	24.5–28 cm	28–32 cm	32–36 cm
Calf	26.5–37 cm	29–39.5 cm	32–43 cm	34.5–47 cm	37–49.5 cm

VENOSAN® Silverline Lady Shoe Size Chart

For persons with large calf muscles, it is recommended that the next larger size be used

	Small	Medium	Large	X-Large	XX-Large
US	4–5	5½–7½	8–10½	11–13	14–15

VENOSAN®

LEGLINE 20

Support Stockings 140 den

ELEGANT LUXURY

Elegant appearance for your legs, tangible prophylaxis for your health! That is VENOSAN®LEGLINE®, the effective compression stockings and support socks with an extremely fine mesh.

SPECIFICALLY DESIGNED

- Therapeutically correct, graduated compression
- Sheer good looks and superior comfort fit
- Fashionable colours




VENOSAN® Legline 20 Size Chart (cm)

For an accurate fit to your exact measurements, use the chart below


	X-Small	Small	Medium	Large	X-Large
Ankle	20–21 cm	22–23 cm	24–25 cm	26–27 cm	28–29 cm
Calf	31–33 cm	34–36 cm	37–39 cm	40–42 cm	43–45 cm
Below Knee	29–31 cm	32–34 cm	35–37 cm	38–40 cm	41–43 cm
Mid-thigh	42–44 cm	45–47 cm	48–50 cm	51–53 cm	54–60 cm

AD & AGH

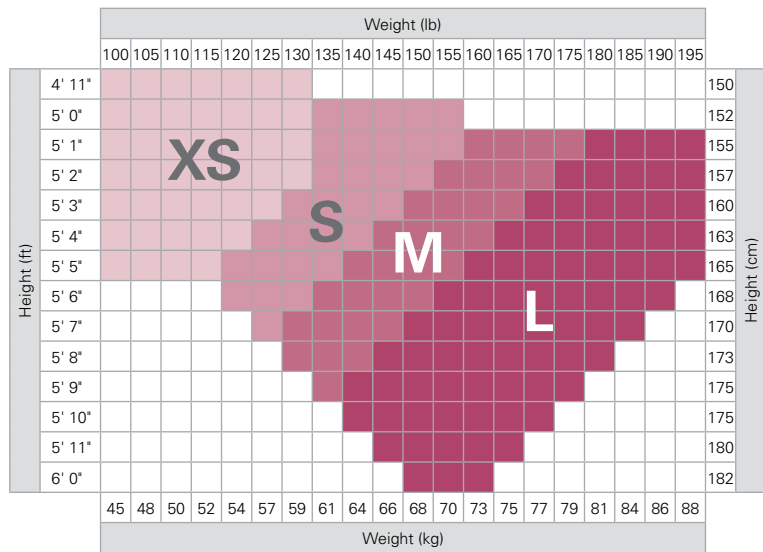

XS X-SMALL
S SMALL
M MEDIUM
L LARGE

AT & ATM

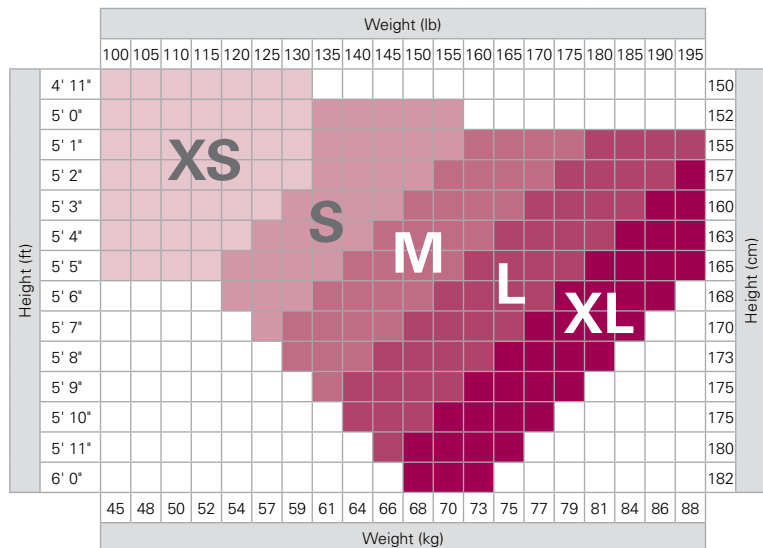



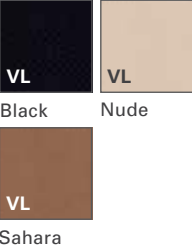

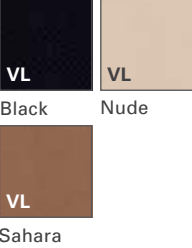

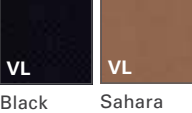

XS X-SMALL
S SMALL
M MEDIUM
L LARGE
XL X-LARGE (AT/ATM ONLY)

VENOSAN® Legline 20

SUPPORT
CLASS I

Pressure at the ankle
20 mmHg

Material: 65% Nylon,
35% Lycra Elastine

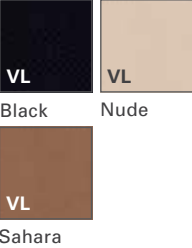Colour		Article	Colour	Toe	X-Small	Small	Medium	Large	X-Large
 VL Black Nude Sahara	 AD Below Knee	Black	closed	303BL	304BL	305BL	306BL		
		Nude	closed	303NU	304NU	305NU	306NU		
		Sahara	closed	303SA	304SA	305SA	306SA		
 VL Black Nude Sahara	 AGH Thigh High Lace top	Black	closed	323BL	324BL	325BL	326BL		
		Nude	closed	323NU	324NU	325NU	326NU		
		Sahara	closed	323SA	324SA	325SA	326SA		
 VL Black Nude Sahara	 AT Pantyhose	Black	closed	333BL	334BL	335BL	336BL	337BL	
		Nude	closed	333NU	334NU	335NU	336NU	337NU	
		Sahara	closed	333SA	334SA	335SA	336SA	337SA	
 VL Black Sahara	 ATM Maternity	Black	closed	343BL	344BL	345BL	346BL	347BL	
		Sahara	closed	343SA	344SA	345SA	346SA	347SA	

VENOSAN®

LEGLINE 30

Support Stockings 280 den

ELEGANT LUXURY

Elegant appearance for your legs, tangible prophylaxis for your health! That is VENOSAN®LEGLINE®, the effective compression stockings and support socks with an extremely fine mesh.

SPECIFICALLY DESIGNED

- Therapeutically correct, graduated compression
- Sheer good looks and superior comfort fit
- Fashionable colours




VENOSAN® Legline 30 Size Chart (cm)


For an accurate fit to your exact measurements, use the chart below

	X-Small	Small	Medium	Large	X-Large
Ankle	20–21 cm	22–23 cm	24–25 cm	26–27 cm	28–29 cm
Calf	31–33 cm	34–36 cm	37–39 cm	40–42 cm	43–45 cm
Below Knee	29–31 cm	32–34 cm	35–37 cm	38–40 cm	41–43 cm
Mid-thigh	42–44 cm	45–47 cm	48–50 cm	51–53 cm	54–60 cm

AD AGH

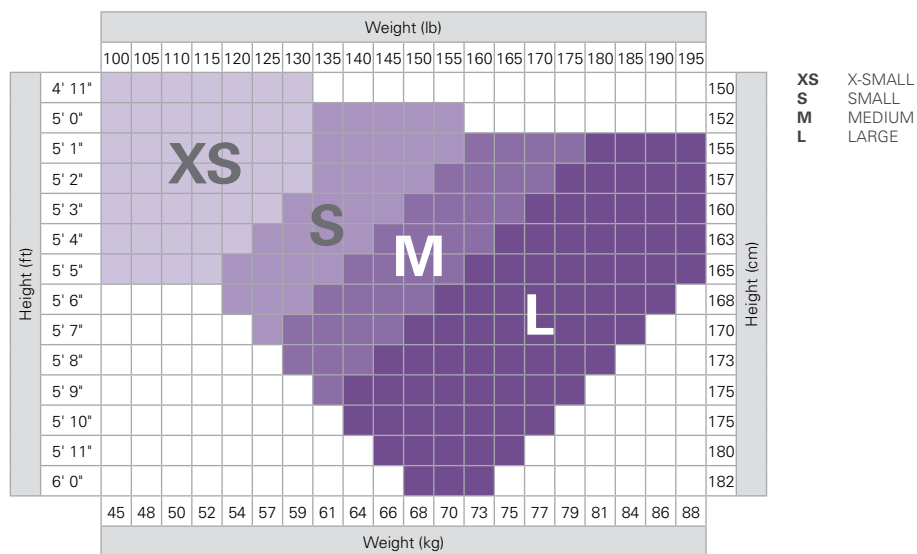

AT

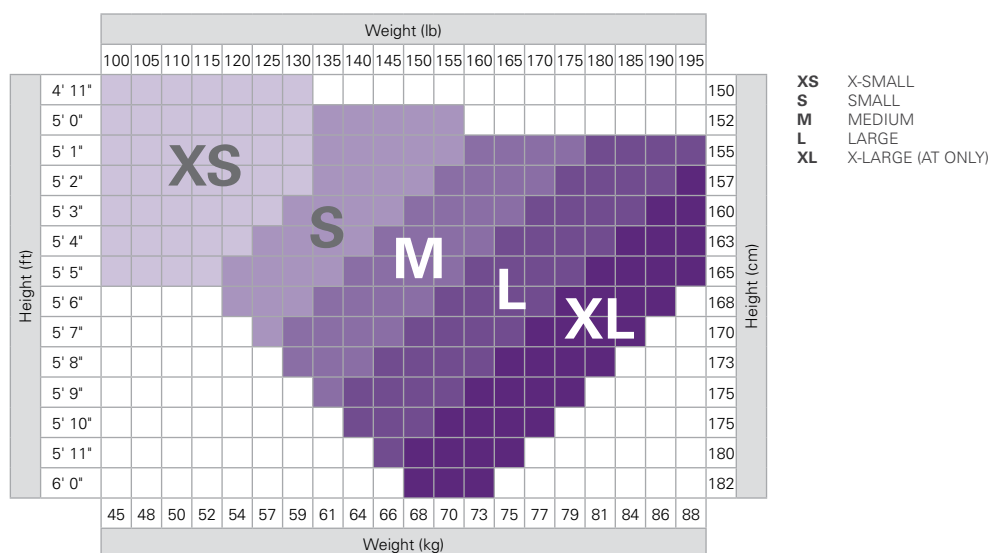


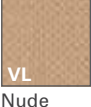




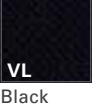
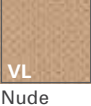

VENOSAN® Legline 30

SUPPORT
CLASS

Pressure at the ankle
30 mmHg

Material: AD, AGH - 65% Nylon, 35% Elastane (Latex Free)
AT - 28% Elastane, 72% Polyamid (Latex Free)

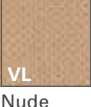

Colour		Article	Colour	Toe	X-Small	Small	Medium	Large	X-Large
 VL Black	 VL Nude	AD Below Knee 	Black	closed	403BL	404BL	405BL	406BL	
			Nude	closed	403NU	404NU	405NU	406NU	
			Sahara	closed	403SA	404SA	405SA	406SA	
 VL Black	 VL Nude	AGH Thigh High Lace Top 	Black	closed	423BL	424BL	425BL	426BL	
			Nude	closed	423NU	424NU	425NU	426NU	
			Sahara	closed	423SA	424SA	425SA	426SA	
 VL Black	 VL Nude	AT Pantyhose 	Black	closed	433BL	434BL	435BL	436BL	437BL
			Nude	closed	433NU	434NU	435NU	436NU	437NU
			Sahara	closed	433SA	434SA	435SA	436SA	437SA

VENOSAN®

ATHLETIC

Unisex Compression Socks

PERFORMANCE SOCKS FOR MEN AND WOMEN

Designed for everyday wear and broad gender appeal. Whether the need for athletic, travel or daily living, you will be excited these compression therapy options exist. Cushioned foot beds and balloon toe pockets, provide day – long benefits of comfort and relief from the effects of venous disease and / or daily living.

INDICATIONS

Everyday wear, travel, athletic need, mild oedema, prevention and treatment of mild venous symptoms, heaviness and fatigue in the legs.

BENEFITS

- Unisex compression socks
- Reciprocated Heel and toe for outstanding fit and comfort
- Seamless toe, ventilated foot for breathability and arch support
- Cushioned achilles tendon area for better protection and comfort

MATERIAL

75% Nylon, 12% Polyester, 13% Spandex.
Made in the USA.



VENOSAN® Athletic Unisex Size Chart (cm)







For an accurate fit to your exact measurements, use the chart below

SUPPORT
CLASS

Pressure at the ankle
20-30 mmHg

Material: 75% Nylon, 12% Polyester,
13% Spandex



Swatch	Article	Colour	Toe	Small	Medium	Large	X-Large
 W White	AD Below Knee 	White	closed	A59202W	A59203W	A59204W	A59205W
 B Black		Black	closed	A59202B	A59203B	A59204B	A59205B
 F Pink		Pink	closed	A59202F	A59203F	A59204F	A59205F
 P Purple		Purple	closed	A59202P	A59203P	A59204P	A59205P
 S Blue		Blue	closed	A59202S	A59203S	A59204S	A59205S

VENOSAN® Athletic Size Chart (cm)

For an accurate fit to your exact measurements, use the chart below

	Small	Medium	Large	X-Large
Calf	29–38 cm	30–42 cm	32–46 cm	34–50 cm
Ankle	18–21 cm	22–25 cm	25–29 cm	29–33 cm

VENOSAN® Athletic Shoe Size Chart

For persons with large calf muscles, it is recommended that the next larger size be used

	Small	Medium	Large	X-Large
Ladies' US	6–8	8½–11	11½–13	13+
Men's US	7 & under	7½–10	10½–12	12½+

VENOSAN®

SPORTSLINE

Compression Support Socks

PERFORMANCE SUPPORT


VENOSAN® SportsLine is a graduated compression support sock for active men and women. Coolmax® draws moisture away from the skin and helps regulate your body temperature to keep you cool.

WHY COOLMAX?

Coolmax® is the original performance fabric that's been helping top athletes push harder and longer but is now being discovered for the cool comfort it brings to everyday apparel.

Coolmax® has proven physiological benefits that, depending on your needs, can be applied to or found in a whole new range of clothing styles – from moderate and high-intensity performance apparel to everyday casual wear.

So consumers get the level of performance they need to stay cool and comfortable in any situation.

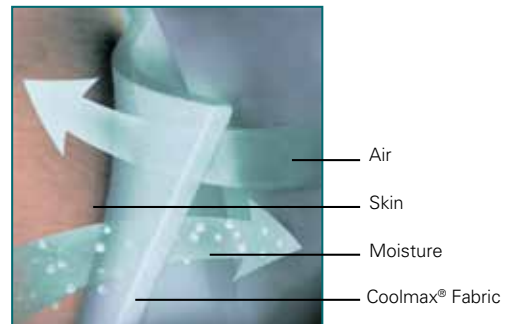


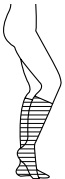

VENOSAN® SportsLine Unisex

SUPPORT
CLASS

Pressure at the ankle
15-20 mmHg

Material: 65% Coolmax®, 26% Nylon,
9% Spandex (Latex Free)



Swatch	Article	Colour	Toe	Size III	Size IV	Size V	Size VI
 SP White	AD Below Knee 	White	closed	69202	69203	69204	69205
 SP Black		Black	closed	89202	89203	89204	89205

VENOSAN® SportsLine Size Chart (cm)

For an accurate fit to your exact measurements, use the chart below

	Small	Medium	Large	X-Large
Calf	28.6–38 cm	30.5–42 cm	31.8–46 cm	34–50cm
Ankle	17.8–21 cm	22.2–25.5 cm	25–29 cm	29–33 cm

VENOSAN® SportsLine Shoe Size Chart

For persons with large calf muscles, it is recommended that the next larger size be used

	Small	Medium	Large	X-Large
Ladies' US	6–8	8½–11	11½–13	13+
Men's US	7 & under	7½–10	10½–12	12½+




VENOSAN® SportsLine Unisex

SUPPORT
CLASS

Pressure at the ankle
20–30 mmHg

Material: 65% Coolmax®, 26%
Nylon, 9% Spandex (Latex Free)



Swatch	Article	Colour	Toe	Small	Medium	Large	X-Large
 SP Black		White	closed	79202	79203	79204	79205
 SP White		Black	closed	99202	99203	99204	99205

VENOSAN® SportsLine Size Chart (cm)

For an accurate fit to your exact measurements, use the chart below

	Small	Medium	Large	X-Large
Ankle	18–21 cm	22–25 cm	25–29 cm	29–33 cm
Calf	29–38 cm	30–42 cm	32–46 cm	34–50 cm

VENOSAN® SportsLine Shoe Size Chart

For persons with large calf muscles, it is recommended that the next larger size be used

	Small	Medium	Large	X-Large
Ladies' US	6–8	8½–11	11½–13	13+
Men's US	7 & under	7½–10	10½–12	12½+

VENOSAN®

AES

Anti Embolism Stockings

THROMBOSIS AND EMBOLISM PREVENTION

The anti-embolism stockings are very easy to put on and offer a high level of comfort in wear due to their breathability.

PERFECT FIT

Anatomical shaping and comprehensive choice of sizes.

COLOUR-CODED

Allows easy and reliable size recognition after washing.

SAFE

Because of the top inspection window.








VENOSAN® AES

SUPPORT
CLASS

Pressure at the ankle
18 mmHg

Material:
90% Nylon Polyamide, 10% Spandex (Latex Free)

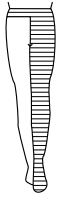

Article & Swatch	Ankle Circumference	Calf Circumference	Thigh Circumference	Length	Size	Size colour code (toe)	Length colour code (top)	Art. No	
AD Below knee 	18–20 cm	to 32 cm		to 40.5 cm	S regular	yellow	natural	SG57000	
				over 40.5 cm	S long	yellow	blue	SG57001	
	20–22 cm	32–37 cm		to 43 cm	M regular	natural	natural	SG57002	
				over 43 cm	M long	natural	blue	SG57003	
	23–25 cm	37–42 cm		to 45.5 cm	L regular	blue	natural	SG57004	
				over 45.5 cm	L long	blue	blue	SG57005	
 SG White	25–28 cm	42–47 cm		to 45.5 cm	XL regular	green	natural	SG57006	
				over 45.5 cm	XL long	green	blue	SG57007	
	28–30 cm	47–52 cm		to 45.5 cm	XXL regular	purple	natural	SG57008	
				over 45.5 cm	XXL long	purple	blue	SG57009	
	AG Thigh Length 	18–20 cm	to 32 cm	45–64 cm	to 73.5 cm	S short	yellow	yellow	SG57020
					73.5–78.5 cm	S regular	yellow	natural	SG57021
over 79 cm					S long	yellow	blue	SG57022	
20–22 cm		32–37 cm	53–71 cm	to 73.5 cm	M short	natural	yellow	SG57023	
				73.5–78.5 cm	M regular	natural	natural	SG57024	
				over 79 cm	M long	natural	blue	SG57025	
23–25 cm	37–42 cm	61–80 cm	to 73.5 cm	L short	blue	yellow	SG57026		
			73.5–78.5 cm	L regular	blue	natural	SG57027		
			over 79 cm	L long	blue	blue	SG57028		
 SG White	25–28 cm	42–47 cm	70–84 cm	to 78.5 cm	XL regular	green	natural	SG57029	
				over 79 cm	XL long	green	blue	SG57030	
	AGG Thigh Length with Belt 	18–20 cm	to 32 cm	45–64 cm	to 73.5 cm	S short	yellow	yellow	SG57050
					73.5–78.5 cm	S regular	yellow	natural	SG57051
					over 79 cm	S long	yellow	blue	SG57052
		20–22 cm	32–37 cm	53–71 cm	to 73.5 cm	M short	natural	yellow	SG57053
73.5–78.5 cm					M regular	natural	natural	SG57054	
over 79 cm					M long	natural	blue	SG57055	
23–25 cm	37–42 cm	61–80 cm	to 73.5 cm	L short	blue	yellow	SG57056		
			73.5–78.5 cm	L regular	blue	natural	SG57057		
			over 79 cm	L long	blue	blue	SG57058		
25–28 cm	42–47 cm	70–84 cm	to 78.5 cm	XL regular	green	natural	SG57059		
			over 79 cm	XL long	green	blue	SG57060		



Lohmann & Rauscher

Lymphoedema Compression Stockings, Ready Wraps and Garments



Lohmann & Rauscher

ReadyWrap™

Clinically effective¹ and easy-to-use^{2,3}
adjustable compression wrap



ReadyWrap™
Adjustable compression
garments for patients
with venous and
lymphatic diseases.

ReadyWrap™


Clinically-effective¹ and easy-to-use^{2,3}
adjustable compression wrap

ReadyWrap is an adjustable variable compression wrap that provides patients and their caregivers an easy-to-use solution to support wound healing and oedema management.¹ This wrap provides clinically-effective compression therapy while supporting patient compliance with its unique alternating strapping system. With ReadyWrap, you can be certain your patients will have an effective and comfortable garment that easily adjusts to their limb and donning abilities.

Why choose ReadyWrap

ReadyWrap is better for patients

Easy to use², with colour-coded VELCRO® brand fasteners

- front-fastening short straps aid patient application and readjustment^{2,3}
- overlapping straps minimise user application error and can prevent undue loss of pressure if the patient does not apply the product perfectly¹
- low-profile design, allowing patients to wear their usual footwear⁴
- all products are available in black or beige 



ReadyWrap is clinically effective

Short stretch properties⁵ – Therapeutic working and tolerable lower resting pressure

- positive outcomes for leg ulcers^{2,3,6} and lymphoedema³ patients
- effective in the management of recurrent swelling (oedema) and ulceration⁶
- excellent SSI of between 16 and 23, similar to inelastic bandages⁵
- 50% overlapping straps prevent gapping and can accommodate changing limb circumference to contain and control oedema^{3,6}



ReadyWrap is cost-effective

ReadyWrap can provide cost savings associated with⁷⁻¹¹:

- time saved due to quicker and easier application of the device, in comparison to compression bandages
- reduced use of bandaging materials – ReadyWrap can be washed and reused over a period of six months
- can be easily applied by the patient or their carer
- can reduce the need for clinic visits due to self-care at home




ReadyWrap™

Adjustable compression system for upper limbs

- 1 Pre-shaped curved straps adjust to the limb providing comfort and support
- 2 Inner liner arm sleeve for correct fitting
- 3 Colour coded VELCRO® brand fasteners makes application easy³



Contact your ReadyWrap distributor for Upper Extemity Measurement and Order Forms

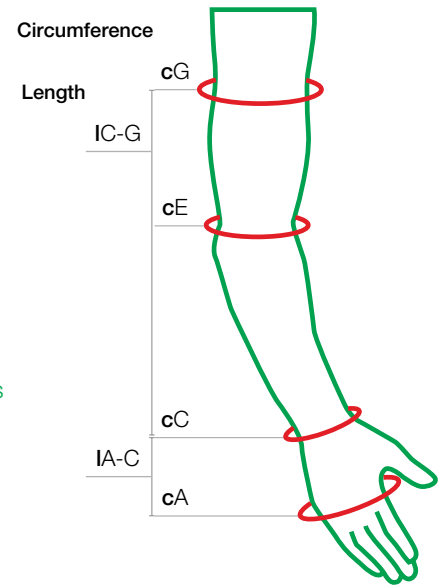

ReadyWrap Arm

- Double-sewn liner allows for easy, one-handed donning and doffing
- Padded wrist and elbow to eliminate compression hot spots and provide additional comfort
- Bilateral design contours to both left and right arms

	Circumference (cm)			Length (cm) (IC-G)	REF	
	Wrist (cC)	Elbow (cE)	Axila (cG)		Beige	Black
Small	13.5-17	19-26	22-29	Short (up to 43cm)	104514	104515
				Average (up to 46cm)	104516	104517
				Tall (up to 49.5cm)	104518	104519
Medium	15.5-20	24-33	26-36	Short (up to 43cm)	104508	104509
				Average (up to 46cm)	104510	104511
				Tall (up to 49.5cm)	104512	104513
Large	17.5-23	29-40	31-43	Short (up to 43cm)	104502	104503
				Average (up to 46cm)	104504	104505
				Tall (up to 49.5cm)	104506	104507
X-Large	17.5-23	30-43	36-50	Short (up to 43cm)	104520	104521
				Average (up to 46cm)	104522	104523
				Tall (up to 49.5cm)	104524	104525



ReadyWrap Gauntlet

- Provides coverage of the wrist and hand
- Left and right specific design contours more naturally to the hand.
- Built-in padding for palm and dorsum helps provide consistent compression over bony prominences
- Pocket design allows for foam pads to provide additional support
- Designed to overlap the ReadyWrap Arm garment to reduce gapping


	Circumference (cm)		Length (cm) (IA-C)		REF	
	Palm (cA)	Wrist (cC)			Beige	Black
Small	18-20	13.5-17.0	up to 12.5	Right	104498	104499
				Left	104490	104491
Medium	20-22	15.5-20.0	up to 13.5	Right	104496	104497
				Left	104488	104489
Large	22-24	17.5-23.0	up to 14.5	Right	104494	104495
				Left	104486	104487
X-Large	24-26	19.5-26.0	up to 15.5	Right	104500	104501
				Left	104492	104493




ReadyWrap™

Adjustable compression system for lower limbs

- 1 Pre-shaped curved straps adjust to the limb providing comfort and support
- 2 Unique wide calf muscle support spine for comfort³, reduces stretch and maintains garment's shape
- 3 Colour coded VELCRO® brand fasteners – assists ease of application³

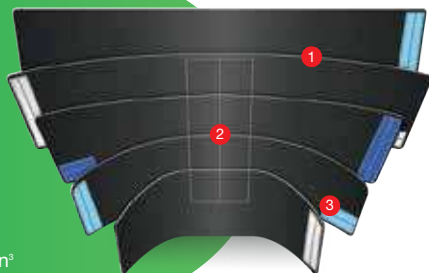

Contact your ReadyWrap distributor for Lower Extemity Measurement and Order Forms

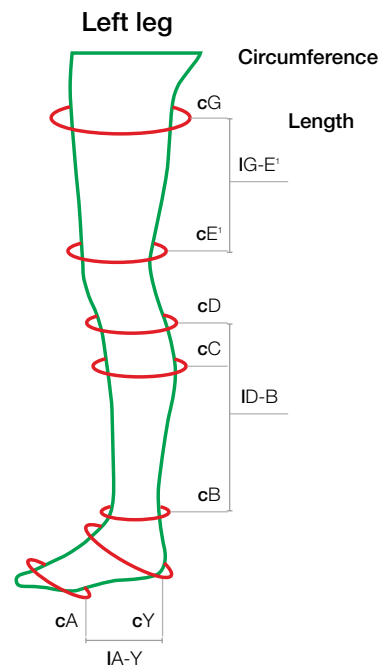

ReadyWrap Toes

- Unique design allows for adjustable toe girths
- Provides full foot coverage when paired with a ReadyWrap Foot unit
- Left and right specific design provides enhanced contouring

	Circumference (cm)		REF	
			Beige	Black
Small	18.0-20.4	Right	104416	104417
		Left	104406	104407
Medium	20.5-23.5	Right	104414	104415
		Left	104404	104405
Large	23.6-26.0	Right	104412	104413
		Left	104402	104403
X-Large	26.1-28.5	Right	104418	104419
		Left	104408	104409
XX-Large	28.6-31.0	Right	104420	104421
		Left	104410	104411



ReadyWrap Foot

- Left and right specific design and medial dart enhance contouring
- Easy to don, two strap design fits inside most shoes
- Overlaps with ReadyWrap Calf and ReadyWrap Toe units to help prevent gapping

	Circumference (cm)		Foot length (cm) (IA-Y)		REF	
	Anterior ankle/heel (cY)	Ankle (cB)			Beige	Black
Small	28-33	18-23	up to 21	Right	104436	104437
				Left	104428	104429
Medium	30-38	20-28	up to 23	Right	104434	104435
				Left	104426	104427
Large	35-43	25-33	up to 26	Right	104432	104433
				Left	104424	104425
X-Large	40-48	30-38	up to 29	Right	104438	104439
				Left	104430	104431



ReadyWrap Foot SL NEW

- Easy to fit in shoe with slim, tubular design
- Regular and Long lengths available to ensure coverage of the foot
- Left and right specific design to enhance contouring and improve fit
- Overlaps ReadyWrap Calf and ReadyWrap Toe units to help prevent gapping

	Circumference (cm)		Foot length (cm) (IA-Y)		REF	
	Foot (cA ¹)	Ankle (cB)			Beige	Black
Small	21-24	18-23	16-19cm (Regular)	Right	144425	144427
				Left	144424	144426
			19-22cm (Long)	Right	144429	144431
				Left	144428	144430
Medium	25-28	19-22	16-19cm (Regular)	Right	144433	144435
				Left	144432	144434
			19-22cm (Long)	Right	144437	144439
				Left	144436	144438
Large	29-32	25-33	16-19cm (Regular)	Right	144441	144443
				Left	144440	144442
			19-22cm (Long)	Right	144445	144447
				Left	144444	144446
X-Large	33-36	30-38	16-19cm (Regular)	Right	144449	144451
				Left	144448	144450
			19-22cm (Long)	Right	144453	144455
				Left	144452	144454



ReadyWrap Calf

- Can be worn with compression hosiery for additional support and protection
- Blocked spine reduces stretch and improves structural support
- Overlaps ReadyWrap Foot and ReadyWrap Knee garments to help prevent gapping

	Circumference (cm)		Length (cm) (IB-D)	REF	
	Ankle (cB)	Calf (cC)		Beige	Black
Small	18-23	28-38	up to 30	104450	104451
			up to 35	104452	104453
Medium	20-28	33-43	up to 30	104446	104447
			up to 35	104448	104449
Large	25-33	38-48	up to 30	104442	104443
			up to 35	104444	104445
X-Large	30-38	43-58	up to 30	104454	104455
			up to 35	104456	104457
XX-Large	35-43	53-68	up to 30	104784	104785
			up to 35	104786	104787



ReadyWrap Knee

- Easy, three-strap design provides a comfortable fit reducing pinching and high pressure points
- Overlaps ReadyWrap Calf and ReadyWrap Thigh to help prevent gapping

	Circumference (cm)		REF	
	Below Knee (cD)	Above Knee (cE ¹)	Beige	Black
Small	28-38	38-48	104464	104465
Medium	33-43	45-58	104462	104463
Large	38-48	52-68	104460	104461
X-Large	43-58	59-78	104466	104467
XX-Large	53-68	66-88	104788	104789



ReadyWrap Thigh

- Low stretch material provides low resting and high working pressures from superior patella border to the groin
- Proximal strap helps accommodate straight and conical thighs
- Overlaps with the ReadyWrap Knee to help prevent gapping

	Circumference (cm)		Length (cm) (IE ¹ - G)	REF	
	Above Knee (cE ¹)	Groin (cG)		Beige	Black
Small	38-48	48-63	up to 25	104478	104479
			up to 30	104480	104481
Medium	45-58	58-73	up to 25	104474	104475
			up to 30	104476	104477
Large	52-68	68-83	up to 25	104470	104471
			up to 30	104472	104473
X-Large	59-78	78-93	up to 25	104482	104483
			up to 30	104484	104485
XX-Large	66-88	88-103	up to 25	104790	104791
			up to 30	104792	104793



ReadyWrap Extender Strap

- Provides an additional 10 cm to a standard ReadyWrap strap
- "Y" clamp design helps ensure a secure fit
- Compatible with ReadyWrap Calf, ReadyWrap Knee and ReadyWrap Thigh

	REF	
	Beige	Black
One Size	304398	304399



ReadyWrap Liners

Select the ReadyWrap Liner which is closest to the equivalent ReadyWrap garment

- Each ReadyWrap lower limb garment (excluding ReadyWrap Toes) is supplied with a ReadyWrap Liner
- Additional liners can be purchased in pairs
- Designed to be used under ReadyWrap garments to protect skin
- Provide no therapeutic compression

For ReadyWrap below knee garments

	REF
	Black
Small	136795
Medium	136796
Large	136797
X-Large	136798

For ReadyWrap Thigh garments

	REF
	Black
Small	136800
Medium	136801
Large	136802
X-Large	136803
XX-Large	136804



ReadyWrap™ UPPER EXTREMITY Measurement and Order Form

How to measure the arm:

Measure the arm in a supported and relaxed position with the elbow slightly flexed. The measurement points refer to the following anatomical landmarks:

Circumference points:

- cA** widest width of the palm, just below the metacarpal phalangeal joints
- cC** narrowest point of the wrist
- cE** around the elbow joint and into the crease of the elbow
- cG** top of the arm, just below the axillary crease

Lengths:


IA-C the length between point **cA** (widest width of the palm, just below the metacarpal phalangeal joints) and point **cC** (narrowest point of the wrist)

IC-G the length between point **cC** (narrowest point of the wrist) and point **cG** (top of the arm, just below the axillary crease)

NB **c** = circumference, **I** = length

All length measurements should follow body contour.
All measurements to be in centimetres (cm).

Write your measurement results in the **box** adjacent to measuring point

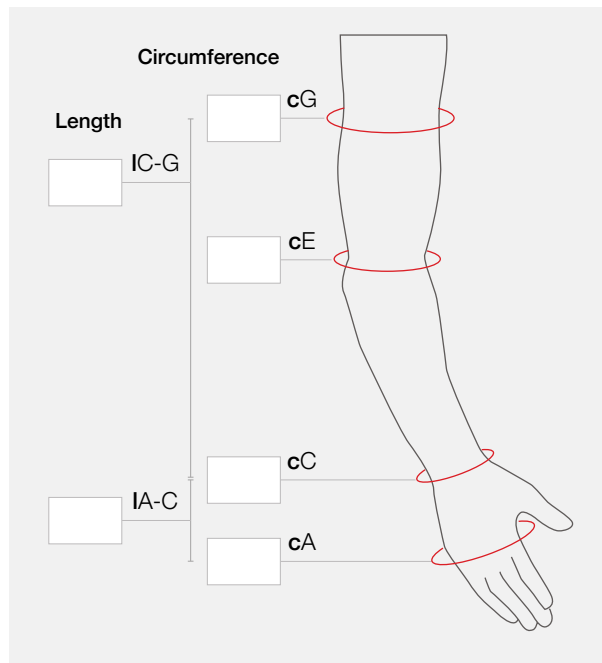

Order Information

Order Date: _____ Patient Name: _____
 Patient Phone: _____ Patient Email: _____
 Clinician Name: _____ Institution/Organisation: _____
 Clinician Phone: _____ Clinician Email: _____

Delivery

Attn: _____
 Address: _____
 City: _____ State: _____ Postcode: _____

Billing Information

Bill to: _____
 Account #: _____
 Card #: _____
 Exp: ____ / ____ CCV: _____

Products

ReadyWrap Arm (sold individually)

Colour		Quantity
<input type="checkbox"/> Black	<input type="checkbox"/> Beige	

ReadyWrap Gauntlet (sold individually)

Style		Colour		Quantity
<input type="checkbox"/> Left	<input type="checkbox"/> Right	<input type="checkbox"/> Black	<input type="checkbox"/> Beige	


ReadyWrap™

LOWER
EXTREMITY

Measurement and Order Form

How to measure the leg:

Ask the patient to stand, if possible. Mark and measure the leg as follows:

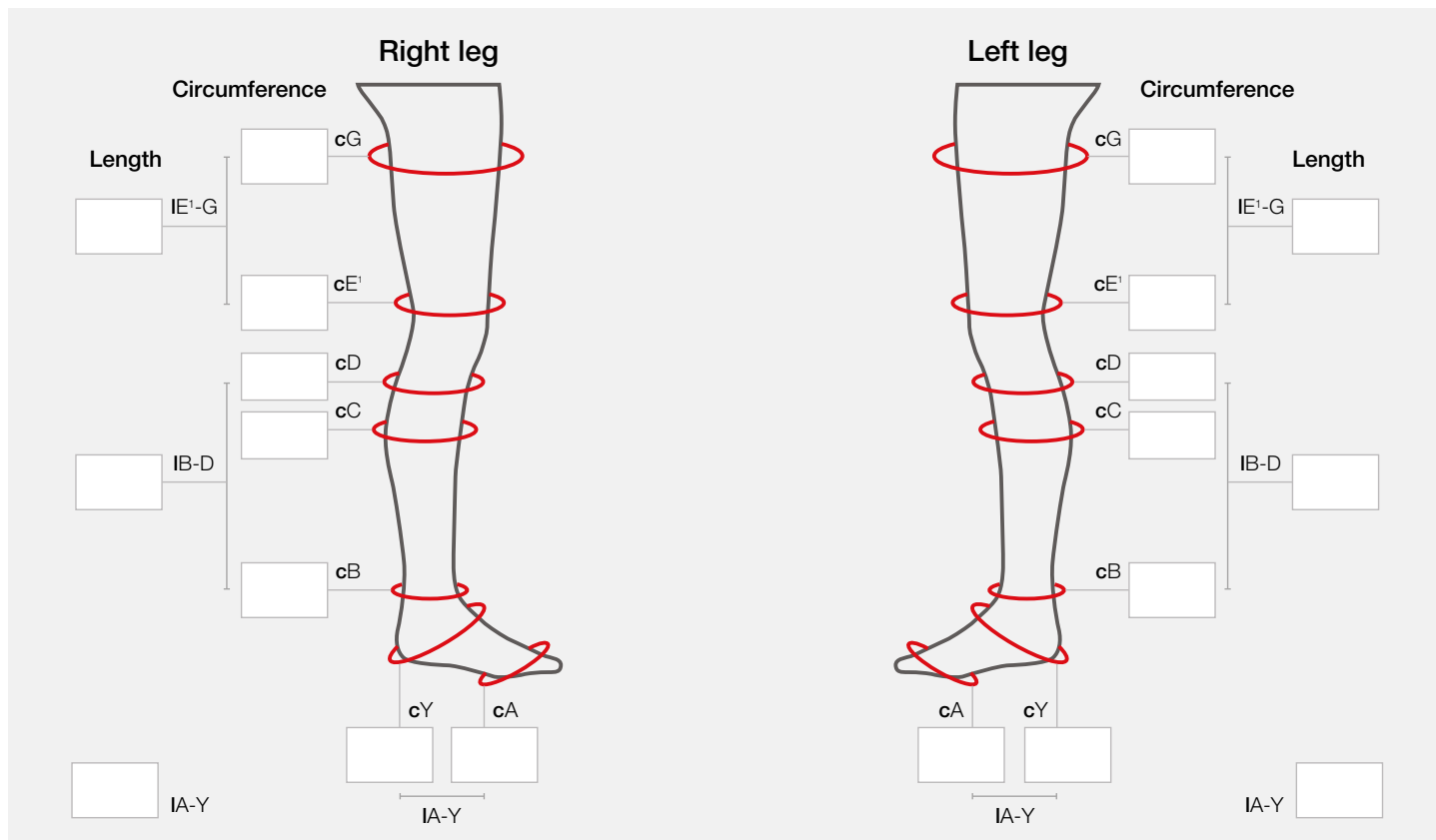

Write your measurement results in the **box** adjacent to measuring point

Measurement points required for each ReadyWrap garment

	Circumference								Length		
	cA	cY	cB	cB	cC	cD	cE¹	cG	IA-Y	IB-D	IE¹-G
ReadyWrap Toe	•										
ReadyWrap Foot & Foot SL		•	•						•		
ReadyWrap Calf				•	•					•	
ReadyWrap Knee						•	•				
ReadyWrap Thigh							•	•			•

Circumference points:

- cG 2 cm below Gluteal Fold
- cE¹ 5 cm above the Knee
- cD 2-3 cm below Popliteal Fossa
- cC widest part of Calf
- cB 2 cm Above Malleolus
- cY around Instep and Heel with foot dorsi-flexed
- cA around base of Toes

Lengths:

- IA-Y heel to base of Toes
- IB-D the length between point cB (2 cm above the malleolus) and point cD (2 cm below popliteal fossa)
- IG-E¹ the length between point cE¹ (5 cm above the knee) and point cG (2 cm below gluteal fold)

NB c = circumference, l = length

All length measurements should be longitudinal and should follow body contours.
All measurements to be in centimetres (cm).

ReadyWrap™

LOWER
EXTREMITY

Measurement and Order Form

Order Information

Order Date: _____ Patient Name: _____

Patient Phone: _____ Patient Email: _____

Clinician Name: _____ Institution/Organisation: _____

Clinician Phone: _____ Clinician Email: _____

Delivery

Attn: _____

Address: _____

City: _____ State: _____ Postcode: _____

Billing Information

Bill to: _____

Account #: _____

Card #: _____

Exp: ____ / ____ CCV: _____

Products

ReadyWrap Toe (sold individually)

Style	Colour	Quantity
<input type="checkbox"/> Left <input type="checkbox"/> Right	<input type="checkbox"/> Black <input type="checkbox"/> Beige	

ReadyWrap Calf (sold individually)

Colour	Quantity
<input type="checkbox"/> Black <input type="checkbox"/> Beige	

ReadyWrap Foot (sold individually)

Style	Colour	Quantity
<input type="checkbox"/> Left <input type="checkbox"/> Right	<input type="checkbox"/> Black <input type="checkbox"/> Beige	

ReadyWrap Knee (sold individually)

Colour	Quantity
<input type="checkbox"/> Black <input type="checkbox"/> Beige	

ReadyWrap Foot SL (sold individually)

Style	Colour	Quantity
<input type="checkbox"/> Left <input type="checkbox"/> Right	<input type="checkbox"/> Black <input type="checkbox"/> Beige	

ReadyWrap Thigh (sold individually)

Colour	Quantity
<input type="checkbox"/> Black <input type="checkbox"/> Beige	

ReadyWrap Liner (sold in pairs)

Style	Colour	Quantity
<input type="checkbox"/> Below Knee <input type="checkbox"/> Thigh High	Black only	

ReadyWrap Extender Strap (sold individually)

Colour	Quantity
<input type="checkbox"/> Black <input type="checkbox"/> Beige	

ExoAnklet (sold in pairs)

Colour	Quantity
Black only	

Submit completed order form to Medical Accessories of Australia.

Email: sales@medicalaccessories.com.au Phone: 1300 527 127 Fax: 07 3870 5944




Lohmann & Rauscher

VENOSAN® Flat

Made-to-measure flat knit
compression garments

Made-to-measure
flat knit compression
garments for the
treatment of Chronic
Oedema and
Lymphoedema

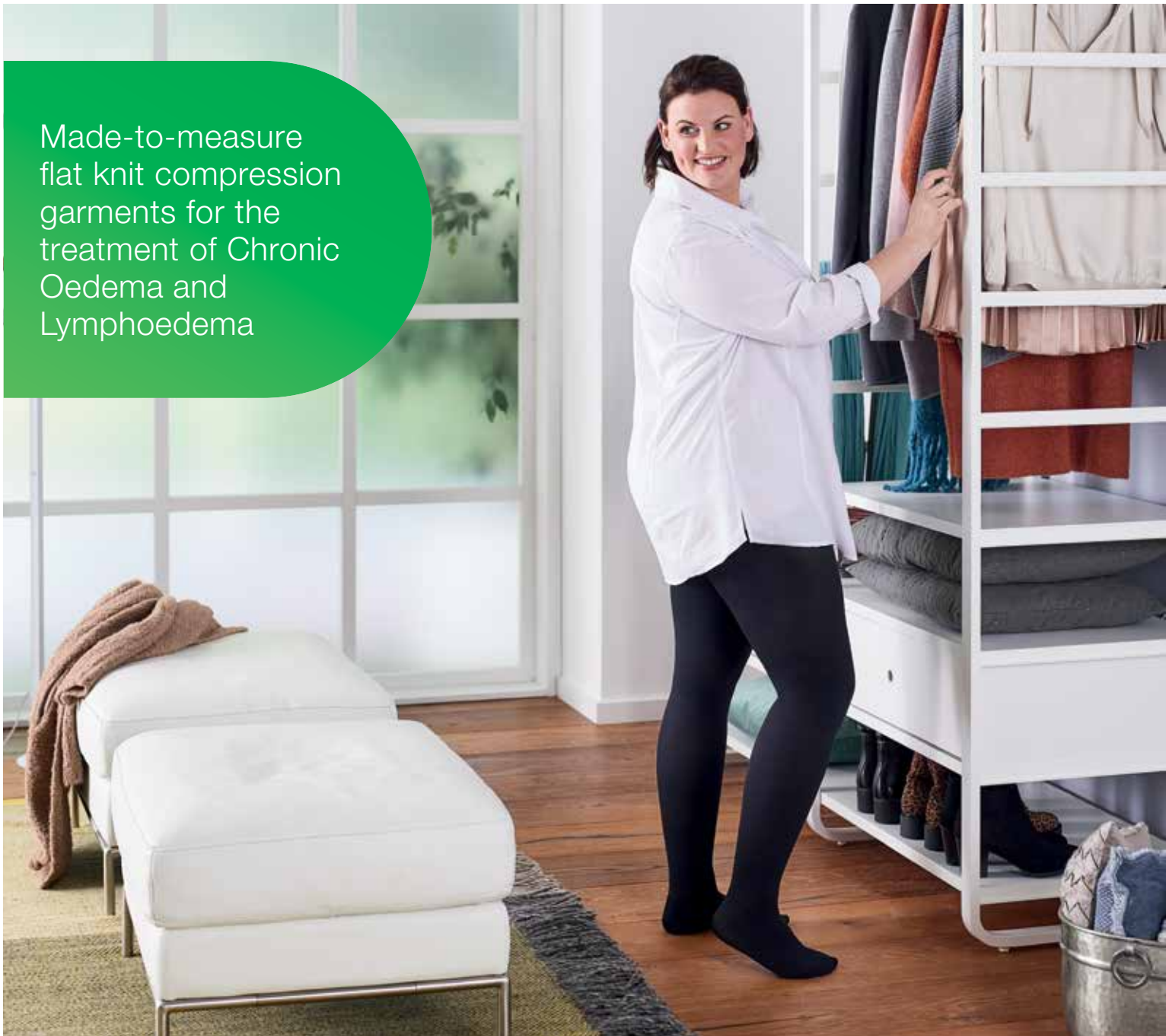

Flat Knit Compression Garments from VENOSAN® Always the right choice.

The addition of VENOSAN Flat to the VENOSAN range, means that we are now able to provide a complete range of compression garments - offering high quality compression solutions for lymphatic & vascular conditions.

Why choose VENOSAN®

- extremely high quality, made-to-measure compression garments
 - design, fabric and manufacturing
 - RAL certified
 - Oeko-Tex Standard 100
- a comprehensive range of upper & lower extremity garments
- 3 types of fabrics – enhanced ability to select the fabric that best suits your patient's needs
 - VENOSAN Soft
 - VENOSAN Pro
 - VENOSAN Strong



The 3 **VENOSAN** Flat fabrics – ability to select the most appropriate fabric for your patient

VENOSAN Soft

Providing patient comfort and concordance with a softer fabric – whilst delivering effective compression.

- extremely soft & durable
- easy to don & doff
- latex free
- class 1–3
- low to medium stiffness

VENOSAN Pro

Providing the fabric stiffness ideal for containing chronic oedema and lymphoedema, whilst maintaining patient comfort and concordance.

- smooth & durable
- increased containment
- easy to don & doff
- class 2 & 3
- medium stiffness

VENOSAN Strong

Providing patients with the highest in fabric stiffness, ideal for patients who are requiring high level of containment.

- smooth & durable
- relatively breathable
- highest containment
- latex free
- class 3 & 4
- high stiffness

VENOSAN® Flat

When to use which product?

Stiffness	Class 1 18–21 mmHg	Class 2 23–32 mmHg	Class 3 34–46 mmHg	Class 4 > 49 mmHg
Low	VENOSAN Soft	VENOSAN Soft		
Medium		VENOSAN Soft & Pro	VENOSAN Soft & Pro	
High			VENOSAN Strong	VENOSAN Strong



VENOSAN® Flat

3 fabrics across 4 compression classes - ability to select the most suitable product to your patient

VENOSAN® Soft 1 & 2

- when a class 1 or 2 soft stocking with low stiffness is indicated
- with fragile skin due to age or medication prescribed
- in case of Latex allergy or sensitivity
- with little or no oedema

VENOSAN® Pro 2

- when a class 2 firm stocking with medium stiffness is indicated
- with 'normal' skin and normal class 2 indications
- with some oedema

VENOSAN® Soft 3

- when a class 3 soft stocking with medium stiffness is indicated
- with fragile skin due to age or medication prescribed
- with little to moderate oedema
- in case of latex allergy or sensitivity

VENOSAN® Pro 3

- when a class 3 firm stocking with a little more than medium stiffness is indicated
- with 'normal' skin and normal class 3 indications
- with moderate oedema

VENOSAN® Strong 3

- when a class 3 stocking with high stiffness is indicated
- serious indications
- standard option: T-heel
- in case of Latex allergy or sensitivity

VENOSAN® Strong 4

- when a class 4 firm stocking with very high stiffness is indicated
- 'Heavy' worker for very serious indications
- standard option: T-heel
- in case of Latex allergy or sensitivity

VENOSAN® Flat

Compression Garments

for lower extremity.



AB
Anklet



AD
Below Knee



AF
Mid Thigh



AG
Thigh High



AGT
Thigh High with
Waist Attachment



AT
Pantyhose



ATS
Single Leg
Pantyhose, short
side finishing at F
or G



BD
Ankle to Knee



BT
Ankle to Waist



DF
Knee



DG
Knee to Thigh



DGT
Knee to Thigh
with Waist
Attachment



DT
Knee to Waist
(Capri)



DTS
Single Leg Capri,
short side
finishing at point
F or G














FT
Bermuda
Pantyhose



FTGT
Single Leg
Bermuda
Pantyhose



The custom-made compression products are characterized by special reinforcements and special processing at joints and cuffs.

	Class 1 18–21 mmHg	Class 2 23–32 mmHg	Class 3 34–46 mmHg
VENOSAN® Soft			
Colours	 	 	 
Stiffness	Low	Low	Medium
		Class 2 23–32 mmHg	Class 3 34–46 mmHg
VENOSAN® Pro			
Colours	 		 
Stiffness	Medium		Medium
		Class 3 34–46 mmHg	Class 4 & 4S > 49 mmHg
VENOSAN® Strong			
Colours			
Stiffness	High		High

Options & Modifications available

- Closed toe or open toe
- Straight or oblique toe
- T-Heel
- Oblique G finish
- Curved Knee
- Horizontal Fly
- Open Crotch
- Silicone band -3cm or 5cm
- Elastic band - 5cm



Flat
Knit
THERAPY



VENOSAN®

Compression Stockings

VENOSAN® flat knit

Measuring Instructions - Lower Extremity

NOTE:

- All measurements in cm
- **c** = circumference, **l** = length
- Unless specified, all length measurements should be longitudinal and should not follow body contours

Leg Measurements

Measuring points B-D can be done with patient sitting or lying.
Above E to be measured with patient standing.

- cG** Two fingers/4cm below Gluteal Fold for circumference
- IG** Two fingers/4cm below Gluteal Fold to ground for length
- cF** Mid Thigh for circumference
- IF** Mid Thigh to ground for length
- cE** Top of kneecap for circumference to avoid being tight at knee pit area
- IE** Middle of Kneecap to ground for length
- cD** Two fingers/4cm below Popliteal Fossa for circumference
- ID** Two fingers/4cm below Popliteal Fossa to ground for length
- cC** Widest part of Calf for circumference
- IC** Widest part of Calf to ground for length
- cB1** Where Calf starts for circumference
- approximately 6-8cm above point B
- IB1** Where Calf starts to ground for length
- cB** Smallest part of the ankle, above Malleolus for circumference
- IB** Smallest part of the ankle, above Malleolus to ground for length
- cY** Around Instep and Heel with foot dorsiflexed for circumference

Foot Measurements

- cA** Around base of toes for circumference.
Always in standing position, weight bearing.
- IA** **Horizontal finish:** Back of heel to base of toes for desired length
Oblique finish:
 - IA_{medial}** From the back of the heel to the base of large toe
 - IA_{lateral}** From the back of the heel to the base of small toe
- IZ** **Closed Toe:** Back of heel to tip of longest toe (Total length of foot)

Panty Section Measurements

Ask patient to put their hands on their waistline

- cT** Waist circumference
- IT** Waist to ground for length
- cH** Largest circumference around the hips or stomach

Measure the following length along contours of body

- IGT** Waist to 5cm/three fingers below Gluteal Fold
- IK¹T** Front length from centre of waist (T) to 5cm above Gluteal Fold applying gentle pressure
- IK²T** Length at the back from the waist (T) over the buttocks to the Gluteal Fold while applying gentle pressure and following contours

Options

Silicone band: Helps keep garment in place.

Elastic band: makes top edge more firm. Does not prevent garment from slipping down. No length adjustments required for silicone or elastic bands.

T-Heel: can prevent rubbing or a crease indentation on the anterior ankle. Standard option in VENOSAN Strong garments.

Oblique G: Garment is finished on an angle.

Ankle inserts: silicone ankle inserts can be requested. No additional measurements needed.

Curved knee: Knee area knitted in a bent position.

Stocking in 2 parts: No additional measurements needed. The overlap will provide no additional pressure. Overlap is 7cm in length, if you do not state otherwise.

Please send the completed Order Form
to Medical Accessories via:
Email: sales@medicalaccessories.com.au
Fax: (07) 3870 5944
Phone: 1300 527 127



Flat
Knit
THERAPY

Send completed form to Medical Accessories of Australia:
Email: sales@medicalaccessories.com.au
Fax: (07) 3870 5944
Phone: 1300 527 127



Lower Extremity Order Form

Date:


Therapist name:	Therapist phone:	Therapist email:
Patient name:	Patient phone:	Patient schema no:
Delivery details:		

Measurements

Hip/Waist	Circumference (cm)	Length (cm)
cT		IGT
cH		IK1T
		IK2T

Foot/Leg

Left circumference (cm)	Length (cm)	Right circumference (cm)
cG	IG	cG
cF	IF	cF
cE	IE	cE
cD	ID	cD
cC	IC	cC
cB1	IB1	cB1
cB	IB	cB
cY		cY
cA	IA	cA
	IZ	

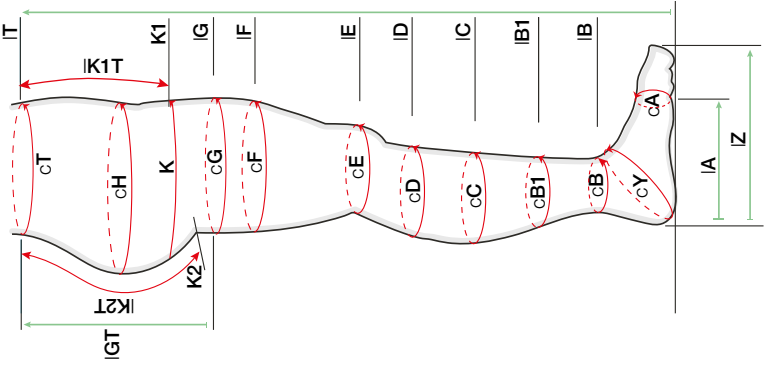

Office use only:

Garment types:

AB AD AG AGT AT AT* BT DT DTS* FT

*short leg can finish at F or G
Additional garment styles are available. Contact Medical Accessories for details.

Product	Col.	Garment type	Qty. (pcs.)	Colour
VENOSAN Soft	1			beige
VENOSAN Soft	2			black
VENOSAN Pro	2			
VENOSAN Soft	3			
VENOSAN Pro	3			
VENOSAN Strong	3			
VENOSAN Strong	4			
VENOSAN Strong	4S			

Options

Bands

3 cm silicone band
 5 cm silicone band
 5 cm elastic band

Other

open toe
 closed toe (please fill in IA and IZ)
 oblique toe (please fill in IA-medial and IA-lateral)
 T-heel
 horizontal fly
 open crotch

full compression body part
 oblique G **
 curved knee
 stocking in 2 parts (ie. AD + DG)


** With AG and DG only possible in combination with 5 cm silicone or elastic band

Comments:


IA-medial	} must be filled in for oblique toe
IA-lateral	

VENOSAN® Flat

Compression Garments
for upper extremity.

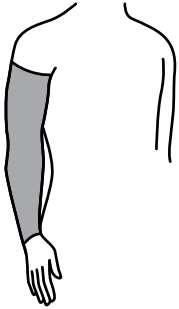

CG
Wrist to Axilla

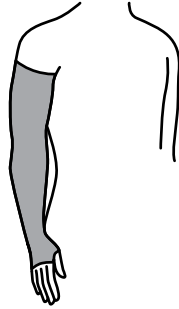







AG
Palm to Axilla







AC1
Gauntlet



VENOSAN® Soft

	Class 1 18-21 mmHg	Class 2 23-32 mmHg	Class 3 34-46 mmHg
Colours	 	 	 
Stiffness	Low	Low	Medium

VENOSAN® Pro

	Class 2 23-32 mmHg	Class 3 34-46 mmHg
Colours	 	 
Stiffness	Medium	Medium

Options & Modifications available

- Silicone band - 3cm
- Silicone band - 5cm
- Oblique G finish



Flat
Knit
THERAPY



VENOSAN®

Compression Stockings

VENOSAN® flat knit

Measuring Instructions - Upper Extremity

NOTE:

- All measurements in cm
- **c** = circumference, **l** = length
- The patient should be seated when measurements are taken

Arm Measurements

Measure the arm with palm facing upwards. Mark the various measuring points on the inside of the arm.

Circumferences

Circumference measurements are taken with a slight bend in the elbow (approx. 150 degrees).

- cC** Narrowest part of the wrist. Measure loosely.
- cD** Mid forearm. The widest part of the arm between points C and E.
- cE** Elbow. Circumference around elbow. Slight bend of approx. 150 degrees.
- cF** Middle of the upper arm. Approximately half way between E and G.
- cG** Upper arm in the axillary fold. This is the top of the garment beneath the axilla. To determine correct location it may be helpful to wrap a magazine or thick piece of paper around the arm, under the axilla. The top of the magazine or paper will be point G.

Lengths

Length measurements should be taken on the inside of an outstretched arm.

- ID** Length from C to D
- IE** Length from C to E
- IF** Length from C to F
- IG** Length from C to G

Options

Silicone band: Helps keep garment in place

Oblique G: Garment is finished on an angle.

Hand Measurements

Fingers should be spread when measuring the hand. Mark points A, B and C on the palm side of the hand in the middle.

Circumferences

- cA** Around MCP heads. The widest part.
- cB** Base of the thumb/thumb web space. Runs parallel to point A.
- cC** Narrowest part of the wrist. Measure loosely.
- cC¹** Record circumference of where gauntlet will finish. Most commonly 6-8cm from C.
- cZ** Circumference at the distal point on the thumb where the thumb covering will end. It is recommended to avoid locating point Z on the IP joint.
- cX** Circumference at the base of the thumb. Take an oblique measurement over the base of the thumb joint.

NOTE: C-C¹ is manufactured with no additional compression

Lengths

- IA-B** Length from A to B ensuring hand is outstretched
- IA-C** Length from A to C ensuring hand is outstretched
- IA-C¹** Length from A to C¹, where the gauntlet will finish. Commonly an additional 6-8cm to IA-C length
- IZ-X** Length from Z to X

Please send the completed Order Form to Medical Accessories via:
Email: sales@medicalaccessories.com.au
Fax: (07) 3870 5944
Phone: 1300 527 127



Flat
Knit
THERAPY

Send completed form to Medical Accessories of Australia:
Email: sales@medicalaccessories.com.au
Fax: (07) 3870 5944
Phone: 1300 527 127




VENOSAN® SOFT
VENOSAN® PRO

Upper Extremity Order Form

Date:

Therapist name: _____ Therapist phone: _____ Therapist email: _____
 Patient name: _____ Patient phone: _____ Patient schema no: _____
 Delivery details: _____ Office use only: _____

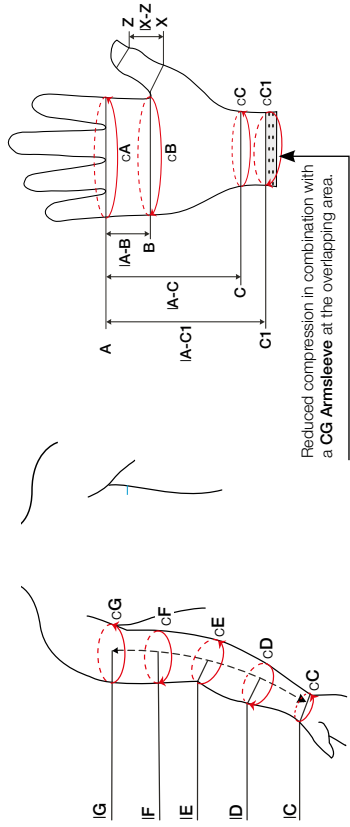

Garment types:

CG Armsleeve
 AC1 Gauntlet
 AG Armsleeve with fixed gauntlet

Measurements

Arm	Left circumference (cm)	Length (cm)	Right circumference (cm)
cG	IG		cG
cF	IF		cF
cE	IE		cE
cD	ID		cD
cC			cC
cA			cA
cB	IA-B		cB
cC	IA-C		cC
cC1	IA-C1		cC1
cZ			cZ
cX	IZ-X		cX

Product	Ccl. type	Garment Qty. (pcs.)	Gauntlet		right	left	3 cm	5 cm	Silicone band	Oblique G	Colour
			apart	fixed							
VENOSAN Soft	1										black
VENOSAN Soft	2										beige
VENOSAN Pro	2										black
VENOSAN Soft	3										beige
VENOSAN Pro	3										black

Comments:

ExoStrong™

Off-the-shelf flat knit compression garments.



ExoStrong™

Affordable alternative to custom garments.

ExoStrong™ is a full line of off-the-shelf flat knit gradient compression garments for the maintenance of lymphoedema or oedema during the day or periods of high activity.

For lymphoedema patients, flat knit garments are one of the primary self-care options. Where circular knit migrates into the skin folds, flat knit helps shape the limb and provides the high containment needed to maintain reductions made in therapy.

ExoStrong is a fraction of the price of custom flat knits — allowing patients to choose the right daytime garment at an affordable price.



Made in the USA

High-quality flat knit, manufactured at our headquarters in Milwaukee, Wisconsin.



Off-the-Shelf

Full line of garments readily available in a wide variety of sizes.



Supports mobility

High containment garment in a comfortable and functional design.



Quick and easy fitting

Only 2–5 measurements are needed to select size for each style.



Easy garment care

Machine washable and dryable to maintain fit for the life of the garment.



Service guarantee

Guaranteed to be free of material and manufacturing defects for the six-month life of the garment.

ExoStrong™

Upper Extremity garments



No Seams
Gauntlet & Glove



ExoStrong™ Gauntlet

sold individually

Compression	20–30 mmHg	
Color	<input type="radio"/> BEIGE	<input type="radio"/> BLACK
Size	Circumference	
	B [°]	C [°]
Small	17–19	15–17
Medium	19–21	17–19
Large	22–24	20–22
X-Large	25–27	23–25

ExoStrong™ Glove

sold individually

Compression	20–30 mmHg		
Color	<input type="radio"/> BEIGE	<input type="radio"/> BLACK	
Finger length	<input type="radio"/> QUARTER	<input type="radio"/> HALF	<input type="radio"/> FULL
Size	Circumference		
	B [°]	C [°]	
Small	17–19	15–17	
Medium	19–21	17–19	
Large	22–24	20–22	
X-Large	25–27	23–25	

ExoStrong™ Arm Sleeve

sold individually

Compression	20–30 mmHg			
Color	<input type="radio"/> BEIGE	<input type="radio"/> BLACK		
Length	<input type="radio"/> SHORT	<input type="radio"/> AVERAGE	<input type="radio"/> TALL	
Top Style	<input type="radio"/> KNIT	<input type="radio"/> SILICONE		
Size	Length	Circumference		
	CG ^L	C [°]	E [°]	G [°]
Small				
Short	up to 43	15–17	24–27	27–30
Average	up to 46			
Tall	up to 49			
Medium				
Short	up to 43	17–19	27–31	31–35
Average	up to 46			
Tall	up to 49			
Large				
Short	up to 43	20–22	31–35	35–40
Average	up to 46			
Tall	up to 49			
X-Large				
Short	up to 43	23–25	36–41	40–45
Average	up to 46			
Tall	up to 49			

All measurements in centimeters.

ExoStrong™

Lower Extremity garments



ExoStrong™ Below Knee

sold individually

Compression	20–30 mmHg
Color	<input type="radio"/> BEIGE <input type="radio"/> BLACK
Length	<input type="checkbox"/> SHORT <input type="checkbox"/> AVERAGE <input type="checkbox"/> TALL
Top Style	<input type="checkbox"/> KNIT <input type="checkbox"/> SILICONE
Toe Style	<input type="radio"/> OPEN

Size	Length		Circumference	
	AY ^L	YD ^L	B [°]	C [°]
Small				
Short	up to 17	up to 38	18–20	27–36
Average	up to 19	up to 41		
Tall	up to 22	up to 44		
Medium				
Short	up to 17	up to 38	20–24	29–39
Average	up to 19	up to 41		
Tall	up to 22	up to 44		
Large				
Short	up to 17	up to 38	24–28	32–43
Average	up to 19	up to 41		
Tall	up to 22	up to 44		
X-Large				
Short	up to 17	up to 38	28–31	33–45
Average	up to 19	up to 41		
Tall	up to 22	up to 44		
XX-Large				
Short	up to 17	up to 38	32–34	37–50
Average	up to 19	up to 41		
Tall	up to 22	up to 44		

ExoStrong™ Thigh High

sold individually

Compression	20–30 mmHg
Color	<input type="radio"/> BEIGE <input type="radio"/> BLACK
Length	<input type="checkbox"/> SHORT <input type="checkbox"/> AVERAGE <input type="checkbox"/> TALL
Top Style	<input type="checkbox"/> SILICONE
Toe Style	<input type="radio"/> OPEN

Size	Length		Circumference		
	AY ^L	YG ^L	B [°]	C [°]	G [°]
Small					
Short	up to 17	up to 71	18–20	27–36	41–55
Average	up to 19	up to 74			
Tall	up to 22	up to 77			
Medium					
Short	up to 17	up to 71	20–24	29–39	45–61
Average	up to 19	up to 74			
Tall	up to 22	up to 77			
Large					
Short	up to 17	up to 71	24–28	32–43	50–66
Average	up to 19	up to 74			
Tall	up to 22	up to 77			
X-Large					
Short	up to 17	up to 71	28–31	33–45	56–71
Average	up to 19	up to 74			
Tall	up to 22	up to 77			
XX-Large					
Short	up to 17	up to 71	32–34	37–50	66–81
Average	up to 19	up to 74			
Tall	up to 22	up to 77			

ExoStrong™

Off-the-shelf flat knit compression garments.

Indications

- Mild to severe lymphoedema
- Oedema

Intended Uses

- Helps maintain reduced volume and shape by supporting extremities
- Management of mild to severe lymphoedema
- Management of mild to severe oedema (non-blood related)

Contraindications

- Arterial insufficiency/degeneration or ABPI < 0.8
- Moderate to severe peripheral arterial disease
- Untreated congestive heart failure
- Untreated/systemic infection (i.e. cellulitis)
- Allergies or sensitivities to materials

Precautions

- Impaired sensation (peripheral neuropathy)
- Cognitive impairment

Care Instructions

Washing

- Machine wash warm on permanent press
- Do not use bleach
- Do not dry clean

Drying

- Tumble dry low
- Do not iron



Material Content

Nylon (polyamide), lyocell, spandex (elastane), silicone*

*ExoStrong garments with Silicone Top only.

Not made with natural rubber latex.



Lohmann & Rauscher

ExoSoft™

Off-the-shelf circular knit
compression garments



ExoSoft™

Comfortable with excellent breathability

ExoSoft™ is a full range of off-the-shelf gradient compression garments that provide comfort with excellent breathability.

- Soft and smooth
- Lightweight, breathable microfibre fabric helps keep patients cool, dry and comfortable
- Easy for patients to don
- Wide bands help reduce rolling and keep garment in place
- Armsleeve and Thigh High available with silicone top band
- Lower Extremity available in 15-20mmHg and 20-30mmHg
- Upper Extremity available in 20-30mmHg

Lower Extremity Garments

ExoSoft™ Below Knee

sold in pairs

Compression	15–20 mmHg	20–30 mmHg
Color	<input type="radio"/> BEIGE	<input type="radio"/> BLACK
Top Style	KNIT	
Toe Style	<input type="radio"/> OPEN	<input checked="" type="radio"/> CLOSED

Size	Length	Circumference	
	YD ^L	B ^C	C ^C
Small	up to 40	18 – 20	27 – 37
Medium	up to 43	20 – 24	29 – 39
Large	up to 45	24 – 28	32 – 43
X-Large	up to 48	28 – 31	35 – 45


ExoSoft™ Thigh High

sold in pairs

Compression	15–20 mmHg	20–30 mmHg
Color	<input type="radio"/> BEIGE	<input type="radio"/> BLACK
Top Style	≡ SILICONE	
Toe Style	<input checked="" type="radio"/> CLOSED	

Size	Length	Circumference		
	YG ^L	B ^C	C ^C	G ^C
Small	up to 73	18 – 20	27 – 37	41 – 56
Medium	up to 76	20 – 24	29 – 39	45 – 61
Large	up to 78	24 – 28	32 – 43	50 – 66
X-Large	up to 81	28 – 31	35 – 45	56 – 71

All measurements in centimeters.

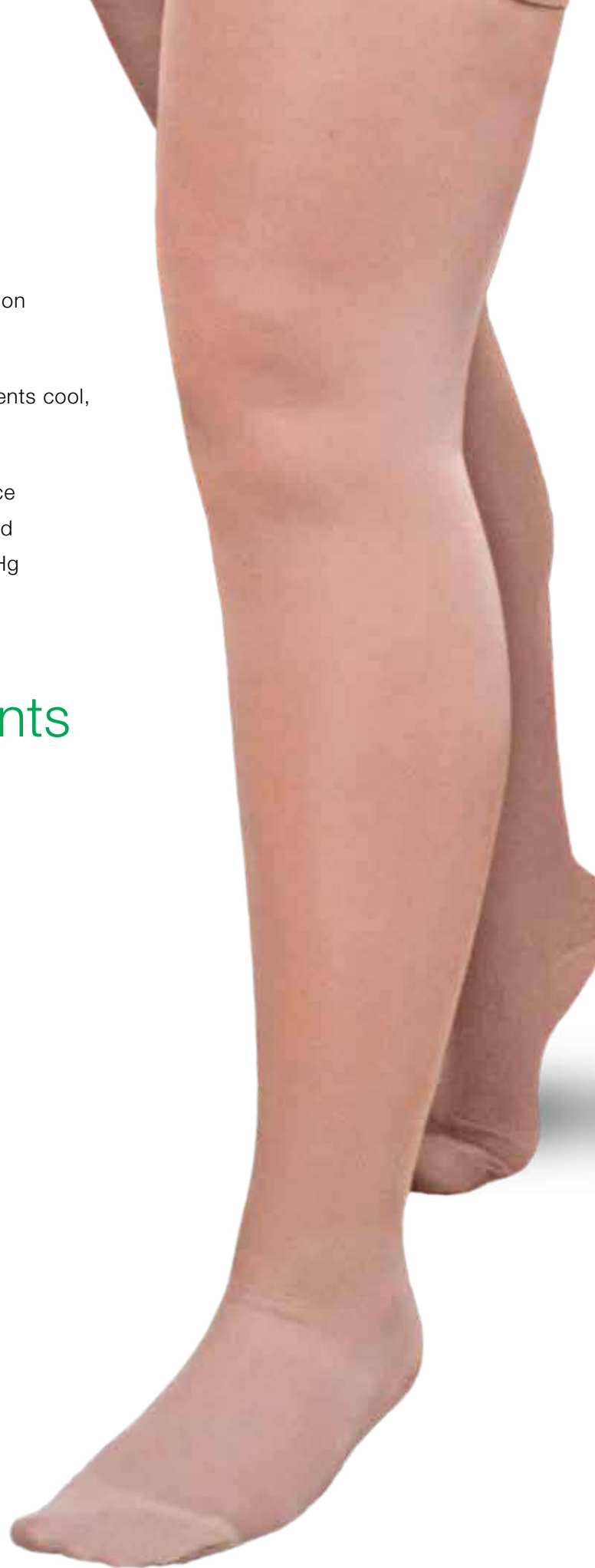

ExoSoft™ Upper Extremity Garments



Pictured:
Quarter Finger
MCPs to Wrist

ExoSoft™ Arm Sleeve

sold individually

Compression	20–30 mmHg
Color	BEIGE BLACK
Length	↓ SHORT ↓ TALL
Top Style	KNIT ▨ SILICONE

Size	Length CG ⁺	Circumference		
		C ^o	E ^o	G ^o
Small				
Short	up to 43	15–17	24–27	27–30
Tall	more than 43			
Medium				
Short	up to 43	17–19	27–31	31–35
Tall	more than 43			
Large				
Short	up to 43	20–22	31–35	35–40
Tall	more than 43			
X-Large				
Short	up to 43	23–25	36–41	40–45
Tall	more than 43			

ExoSoft™ Gauntlet

sold individually

Compression	20–30 mmHg
Color	BEIGE BLACK

Size	Circumference	
	B ^o	C ^o
Small	17–19	15–17
Medium	19–21	17–19
Large	22–24	20–22
X-Large	25–27	23–25

ExoSoft™ Glove

sold individually

Compression	20–30 mmHg
Color	BEIGE BLACK
Finger length	👤 QUARTER 👤 HALF 👤 FULL

Size	Circumference	
	B ^o	C ^o
Small	17–19	15–17
Medium	19–21	17–19
Large	22–24	20–22
X-Large	25–27	23–25

All measurements in centimeters.

ExoSoft™

Off-the-shelf circular knit compression garments

Intended Uses

- Helps maintain reduced volume and shape by supporting extremities
- Management of mild to moderate lymphoedema
- Management of mild to severe oedema (non-blood related)

Contraindications

- Arterial insufficiency/degeneration or ABPI < 0.8
- Untreated congestive heart failure
- Untreated/systemic infection (i.e. cellulitis)
- Allergies or sensitivities to materials

Precautions

- Impaired sensation (peripheral neuropathy)
- Cognitive impairment

Care Instructions

Washing

- Machine wash warm with mild detergent
- Do not use bleach or fabric softeners
- Do not dry clean
- Use of a wash bag is recommended

Drying

- Tumble dry low
- Do not iron



ExoAnklet™

Effective foot compression that easily fits in a shoe.

ExoAnklet™ is a compression anklet that provides focused therapeutic compression on the foot and ankle for the management of lymphedema and edema. This anklet is designed to provide comfortable and effective therapy, while easily fitting into your normal footwear.

- 1 Provides focused therapeutic 18–26 mmHg compression
- 2 Works well on its own, or in combination with other compression garments
- 3 Easily allows for normal footwear with its low-profile design
- 4 Soft, comfortable design, particularly over the toes and heel




ExoAnklet™ compression anklet

sold in pairs

Size	Circumference			Length	Item No.
	MTP (A°)	Ankle (B°)	Heel (Y°)	Foot (Z°)	
Regular					
Small	20–24	18–22	27–33	23–26	136805
Medium	22–26	22–26	29.5–35.5		136806
Large	24–28	26–31	32–38		136807
X-Large	26.5–30.5	31–36.5	34.5–40.5		136808
XX-Large	29–33	36.5–41.5	37–43		136809
Long					
Small	20–24	18–22	27–33	26–29	136810
Medium	22–26	22–26	29.5–35.5		136811
Large	24–28	26–31	32–38		136812
X-Large	26.5–30.5	31–36.5	34.5–40.5		136813
XX-Large	29–33	36.5–41.5	37–43		136814

All measurements in centimeters.

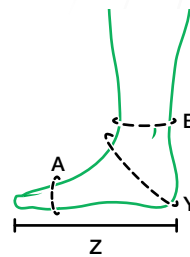

Available in **Black**

Indications

- Mild to moderate lymphedema
- Edema (non-blood related)

Material content

Nylon (polyamide), spandex (elastane)
Not made with natural rubber latex.

Care instructions

- Machine wash on permanent press in warm water
- Do not use bleach or fabric softener
- Do not dry clean
- Lay flat to dry
- Do not iron






Lohmann & Rauscher

TributeWrapTM **NEW**

Ease and comfort within your reach.

Self-adjustable compression foam garment to be used at rest or during the night for the management of lymphoedema or oedema.

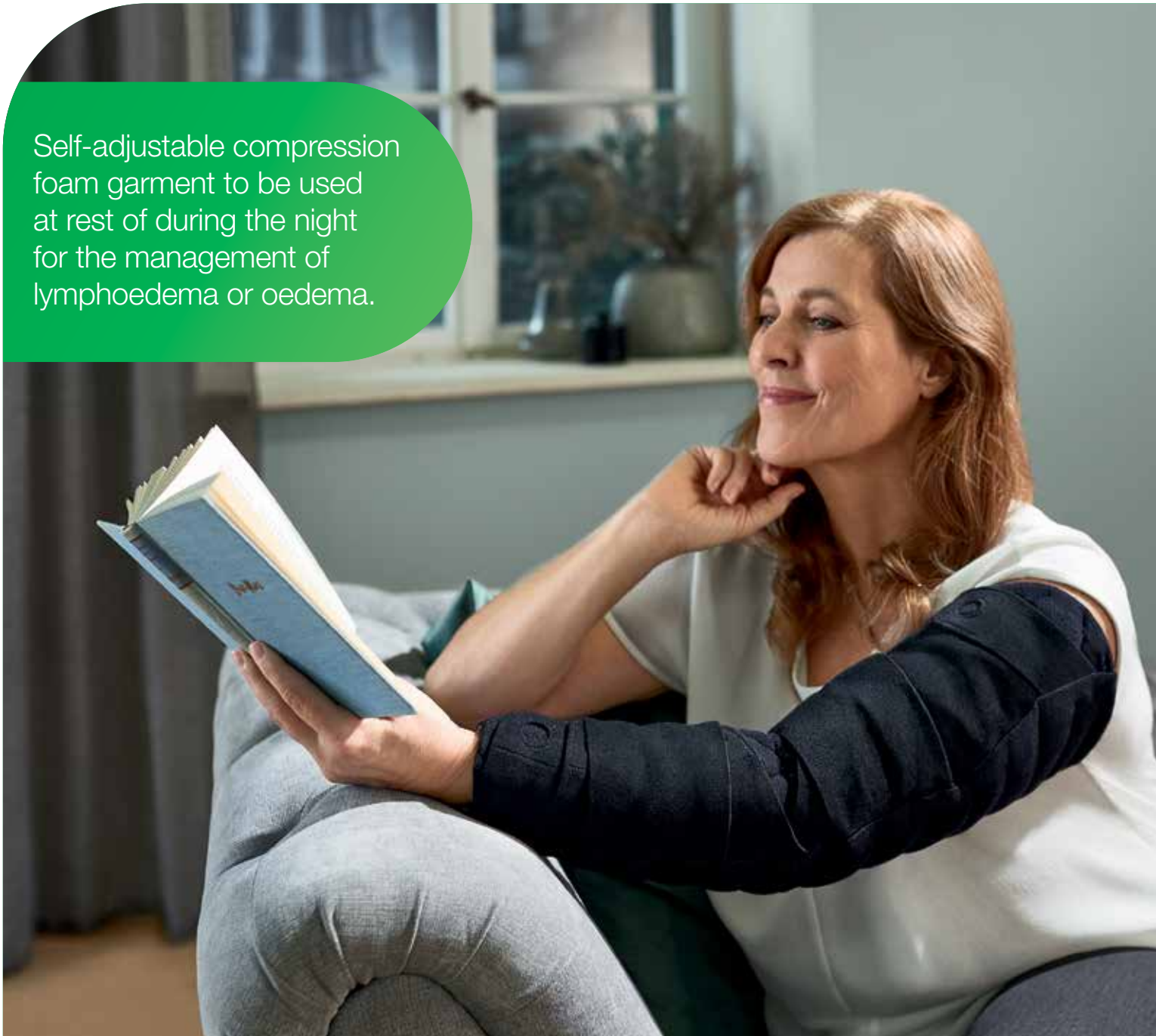

TributeWrap™

Promoting skin health with proven foam technology.

TributeWrap is an adjustable compression garment with foam technology that nestles into the tissue to promote skin health¹. It supports easy self-management of lymphoedema and oedema during low activity by comfortably adjusting to your patient's unique shape and lifestyle.

TributeWrap helps maintain gains made during therapy, while allowing the flexibility to support further reduction. Simply slide it on, adjust the straps, and cover with Sleep Sleeve for comfortable evening compression.



Off-the-Shelf



Evening & Nighttime




Low Activity

[Explore the other great benefits of TributeWrap >](#)



**Easy-to-use
finger grip on
every strap.**

Simple adjustability
for the right fit
and contour.

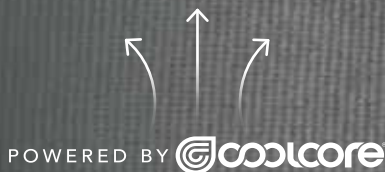

Innovative interior fabric that rapidly wicks away sweat and moisture to regulate body temperature with chemical-free Cooling Power.



Proven foam technology
Breaks up and softens hardened or fibrotic tissue to support skin health.



Sleep Sleeve
Complimentary cover included with all styles to ensure the straps don't get tangled in the sheets.




TributeWrap™ Wrist to Axilla



Chevron channeling

Designed based on MLD principles of directing fluid to collateral pathways.

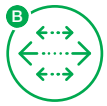

Adjustable strapping system

Easily conforms for the right, comfortable fit, and has flexibility to support further reduction.



Supports mobility

Angled straps at flexion points for a comfortable and functional design.



6-month warranty

TributeWrap is guaranteed to be free of material and manufacturing defects for the six-month life of the garment.



“In the morning, I took it off and couldn’t believe the difference! I wondered if this was a fluke, but I’ve noticed the same results every morning!

This is outstanding! It is easier to put on my daytime flat knit garments, and my legs have less fatigue throughout the day.

I am convinced TributeWrap does make a significant difference!”

John, Lymphoedema patient

TributeWrap™ NEW

Self-adjustable compression foam garment.

Indications



- mild to severe lymphoedema, stage 1–3
- chronic oedema (non-blood related)

Precautions

- impaired sensation (peripheral neuropathy)
- cognitive impairment
- any open wounds should be dressed prior to application

Contraindications

- moderate to severe peripheral arterial disease
- arterial insufficiency or ABPI < 0.8
- untreated congestive heart failure
- untreated infection (e.g. cellulitis)
- allergies or sensitivities to materials
- acute DVT
- severe cognitive impairment

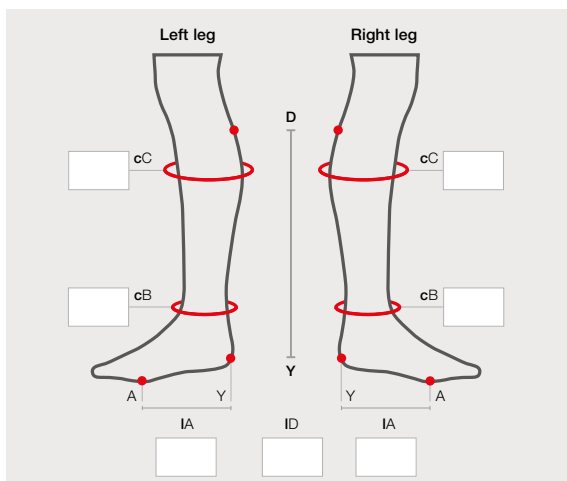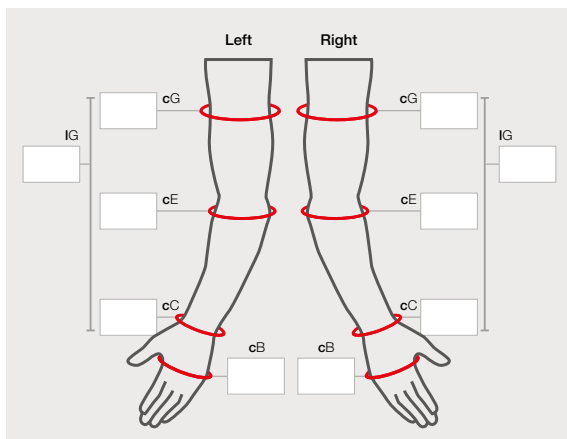

TributeWrap Glove

Self-adjustable compression foam garment for hand and fingers

Size	Circumference (cm)		REF	
	Palm (cB)	Wrist (cC)		
S	16 – 20	16.5 – 21	Left	146 783
			Right	146 786
M	18 – 22	17.5 – 22	Left	146 784
			Right	146 787
L	20 – 24.5	18.5 – 23	Left	146 785
			Right	146 788

~~LATEX~~ Not made with natural rubber latex

TributeWrap Arm

Self-adjustable compression foam garment for wrist to axilla

Size	Circumference (cm)			Length (cm)	REF	
	Wrist (cC)	Elbow (cE)	Axilla (cG)			
S	16 – 21	22 – 28	25 – 31	43 – 47	Left	146 777
					Right	146 780
M	17 – 22	26 – 32	30 – 37	43 – 47	Left	146 778
					Right	146 781
L	18 – 23	30 – 36	35 – 44	43 – 47	Left	146 779
					Right	146 782

~~LATEX~~ Not made with natural rubber latex

TributeWrap Below Knee

Self-adjustable compression foam garment for foot and calf

Size	Circumference (cm)		Length (cm)		REF	
	Ankle (cB)	Calf (cC)	Foot (IA)	Calf (ID)		
S	24 – 30	30 – 36	19 – 22	40 – 44	Left	146 789
					Right	146 792
M	27 – 33	33 – 41.5	19 – 22	40 – 44	Left	146 790
					Right	146 793
L	30 – 36	36 – 45	19 – 22	40 – 44	Left	146 791
					Right	146 794

~~LATEX~~ Not made with natural rubber latex




Lohmann & Rauscher

Caresia™

Off-the-Shelf Bandage Liners

- Caresia™ garments are off-the-shelf bandage liners designed to simplify and enhance compression bandage techniques by replacing the need for traditional padding or under layers.

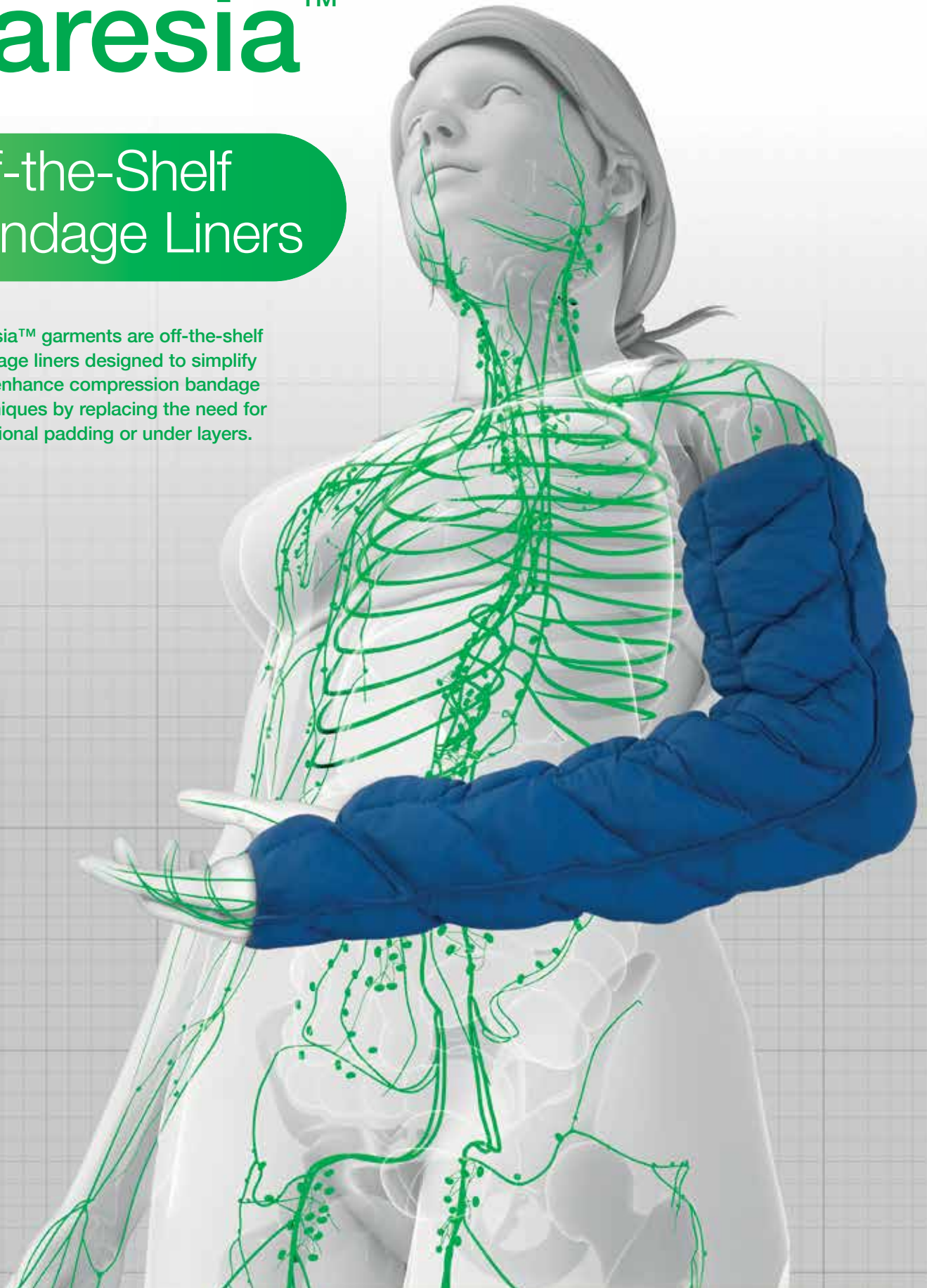

Overview

Why choose Caresia™?

Caresia™ garments are off-the-shelf bandage liners designed by medical professionals to simplify and enhance standard bandaging techniques by eliminating under layers of traditional bandaging. Caresia garments will save you and your patients time, while enhancing therapeutic outcomes.



Reduce time spent bandaging¹ and leave more time for Manual Lymphatic Drainage.



Foam technology helps soften tough fibrotic tissue¹ while remaining gentle on the skin.



Comfortable and wickable fabric.



Machine wash and dry for easy garment care.

Indications

- Lymphoedema, when used with bandages/other garments
- Oedema, (non-blood related) when used with bandages / other garments

Contraindications

- Arterial insufficiency or degeneration / ABPI < 0.8
- Moderate to severe peripheral arterial disease
- Untreated congestive heart failure
- Untreated infection (i.e. cellulitis)
- Impaired sensation (i.e. peripheral neuropathy)
- Severe cognitive impairment
- Allergies or sensitivities to materials
- Not recommended for use on open (i.e. uncovered) wounds



MCP to Axilla

Simply slide on Caresia™ and bandage over top with short-stretch bandages such as Rosidal® K.


Bandages not included.



Caresia™

How to Measure

Measuring for Caresia is simple and easy.

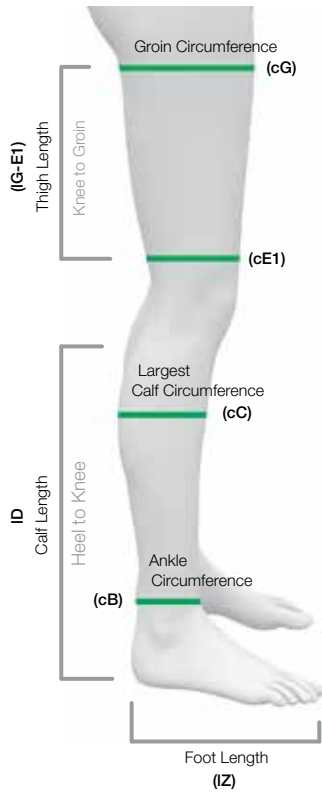

How to measure the Leg:

Ask the patient to stand, if possible. Mark and measure the leg as follows


Circumference points:

- cG 2cm below Gluteal Fold
- cC Widest part of Calf
- cB 2cm Above Malleolus

Lengths:

- IZ Heel to base of Toes
- ID ID The length from point D (2cm below popliteal fossa) to the ground
- IG-E1 The length between point G (2cm below gluteal fold) point E1 (5cm above the Knee measuring from the centre point of the knee cap)

All length measurements should be longitudinal and should follow body contours.

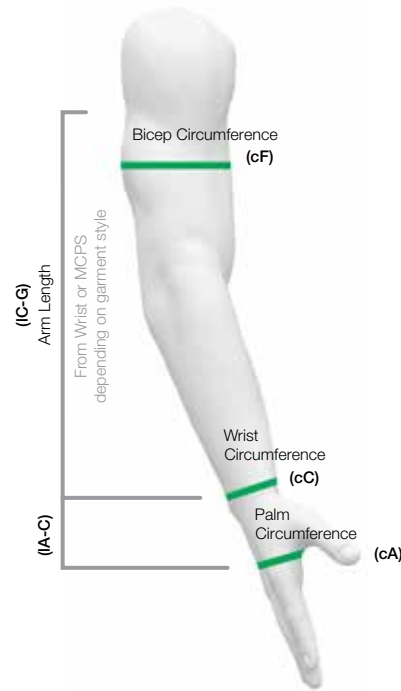

How to measure the arm:

Measure the arm in a supported and relaxed position with the elbow slightly flexed. The measurement points refer to the following anatomical landmarks:

Circumference points:

- cA Widest width of the Palm, just below the metacarpal phalangeal joints
- cC Narrowest point of the Wrist
- cF Widest part of the bicep

Lengths:

- IA-C The length between point A (widest width of the palm, just below the metacarpal phalangeal joints) and point C (narrowest point of the wrist)
- IC-G The length between point C (narrowest point of the wrist) and point G (top of the arm, just below the axillary crease)

Caresia

Upper Extermity Garments

Palm to Axilla

- Provides coverage of the arm from the metacarpals to the axilla, allowing for individual finger wrapping.
- Left / Right specific garments designed with channeling to mimic the collateral pathways.

Description	Arm Length: 59–65cm (IC-G)			REF
	Palm Circumference (cA)	Wrist Circumference (cC)	Biceps Circumference (cF)	
Small (Left)	15–19cm	13.5–17cm	20–30cm	104384
Small (Right)	15–19cm	13.5–17cm	20–30cm	104387
Medium (Left)	19–23cm	17–22cm	30–40cm	104383
Medium (Right)	19–23cm	17–22cm	30–40cm	104386
Large (Left)	23–28cm	22–28cm	40–50cm	104382
Large (Right)	23–28cm	22–28cm	40–50cm	104385



Wrist to Axilla

- Provides coverage of the arm from wrist to the axilla.
- Designed to give patients the option of wearing it as a standalone garment, or combining it with a glove or gauntlet — such as the Caresia Glove or Caresia Gauntlet.
- Left / Right specific garments designed with channeling to mimic the collateral pathways.

Description	Arm Length: 47–53cm (IC-G)		REF
	Wrist Circumference (cC)	Biceps Circumference (cF)	
Small (Left)	13.5–17cm	20–30cm	104390
Small (Right)	13.5–17cm	20–30cm	104393
Medium (Left)	17–22cm	30–40cm	104389
Medium (Right)	17–22cm	30–40cm	104392
Large (Left)	22–28cm	40–50cm	104388
Large (Right)	22–28cm	40–50cm	104391



Glove

- Provides coverage of the hand from the fingertips to wrist.
- Universal fit - can be worn on either left or right hands.
- Designed to be worn as a standalone Glove or to be worn in combination with an arm sleeve — such as the Caresia Wrist to Axilla — for full finger to arm coverage.
- Individual digit spacers come standard.

Description	Length: up to 26cm (IA-C)		REF
	Palm Circumference (cA)	Wrist Circumference (cC)	
Small	15–19cm	13.5–17cm	104378
Medium	19–23cm	17–22cm	104377
Large	23–28cm	22–28cm	104376



Gauntlet

- Provides coverage of the hand from the metacarpals to the wrist.
- Helps allow finger functionality and movement.
- Universal fit - can be worn on either left or right hands.
- Designed to be worn as a standalone Gauntlet or to be worn in combination with an arm sleeve — such as the Caresia Wrist to Axilla — Widest width of the Palm to arm coverage.
- Individual digit spacers come standard.

Description	Length: up to 18cm (IA-C)		REF
	Palm Circumference (cA)	Wrist Circumference (cC)	
Small	15–19cm	13.5–17cm	104381
Medium	19–23cm	17–22cm	104380
Large	23–28cm	22–28cm	104379


Caresia

Lower Extermity Garments

Thigh

- Provides coverage of thigh area from the inferior patella to the groin.
- Designed for flexion with tapered design.
- Available in a wide range of lengths and sizes to accommodate your patients' measurements.
- Combine with a Caresia Below Knee for full leg coverage.

Short	Thigh Length: 28–36cm (IG-E1)		REF
	Groin Circumference (cG)		
Small	50–70cm		104370
Medium	60–80cm		104369
Large	70–90cm		104368
Standard	Thigh Length: 33–41cm (IG-E1)		REF
	Groin Circumference (cG)		
Small	50–70cm		104366
Medium	60–80cm		104365
Large	70–90cm		104364
Tall	Thigh Length: 39–47cm (IG-E1)		REF
	Groin Circumference (cG)		
Small	50–70cm		104374
Medium	60–80cm		104373
Large	70–90cm		104372

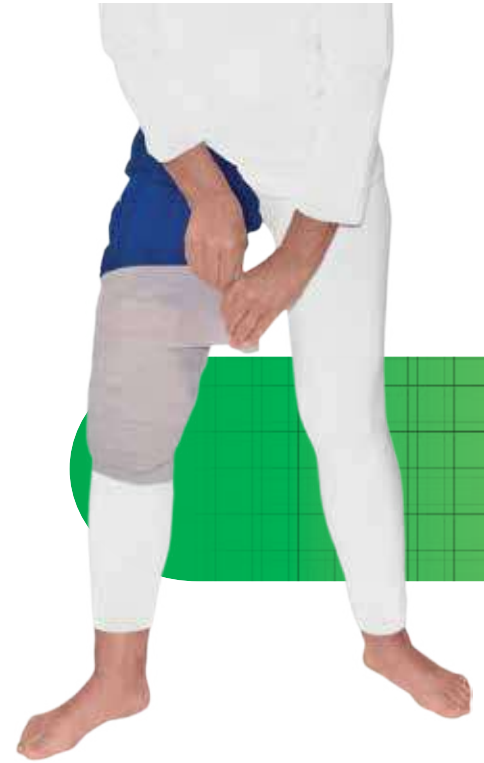

Below Knee

- Provides coverage of the calf from the toes to the inferior patella.
- Available in a wide range of lengths and sizes to accommodate your patients' measurements.
- Combine with a Caresia Thigh garment for full leg coverage.

Short	Calf Length: 34–40cm (ID)		REF
	Ankle Circumference (cB)	Calf Circumference (cC)	
Small	22–28cm	30–40cm	104360
Medium	28–34cm	40–50cm	104359
Large	34–40cm	50–60cm	104358
Standard	Calf Length: 40–46cm (ID)		REF
	Ankle Circumference (cB)	Calf Circumference (cC)	
Small	22–28cm	30–40cm	104357
Medium	28–34cm	40–50cm	104356
Large	34–40cm	50–60cm	104355
Tall	Calf Length: 46–52cm (ID)		REF
	Ankle Circumference (cB)	Calf Circumference (cC)	
Small	22–28cm	30–40cm	104363
Medium	28–34cm	40–50cm	104362
Large	34–40cm	50–60cm	104361



Foot

- Provides closed toe coverage of the foot to the ankle.
- Sold individually.

Description	Foot Length (IZ)	Ankle Circumference (cB)	REF
Small	20–25cm	22–28cm	104354
Medium	25–30cm	28–34cm	104353
Large	30–35cm	34–40cm	104352




SwellSpots™

Oedema and fibrosis pads.



Ideal for difficult areas of oedema, fibrosis, skin folds, misshapened limbs as well as padding and protection.

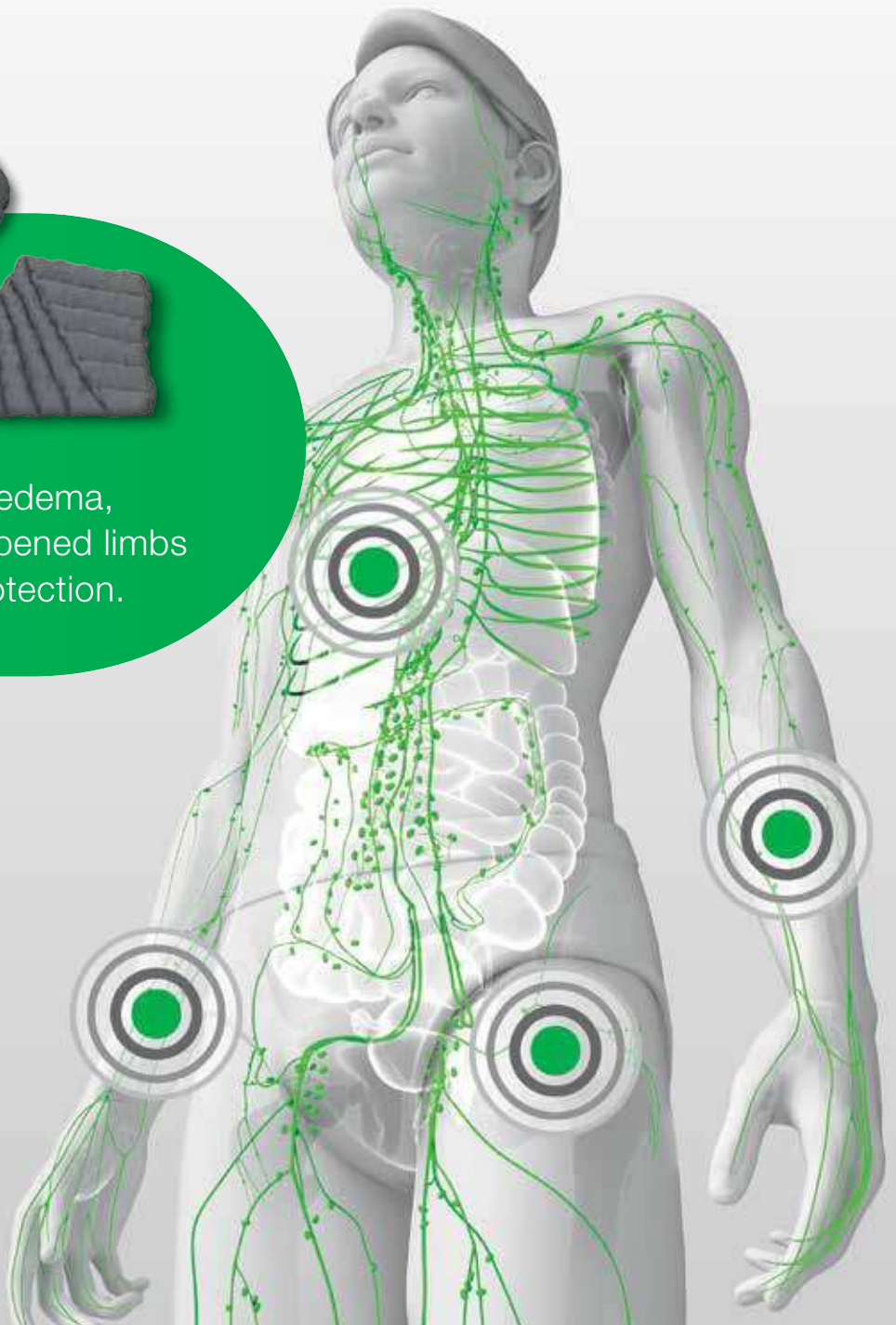

POWERED BY
coolcore

Overview

Why choose SwellSpots™?

POWERED BY


SwellSpots, oedema and fibrosis pads are designed to address particularly troublesome areas by:

- softening tissue fibrosis/hardened skin
- filling concavities
- padding bony prominences
- diffusing pressure points
- protecting vulnerable areas

SwellSpots are available in a wide range of versatile shapes and sizes that tuck into compression garments or under bandages. Powered by Coolcore, these SwellSpots rapidly wick away sweat and moisture from the body, keeping it cool and comfortable.

What is Coolcore?

Coolcore is an innovative, chemical-free fabric that rapidly wicks away sweat and body moisture to regulate body temperature. Its chemical-free technology is built into the fibres of the fabric to withstand its cooling properties through wash and wear during the one year life of the garment.




88%
faster wicking
sweat and moisture
away from the body*



55%
faster
drying time
for easier
garment care*



30%
more
cooling power
to better regulate
body temperature*

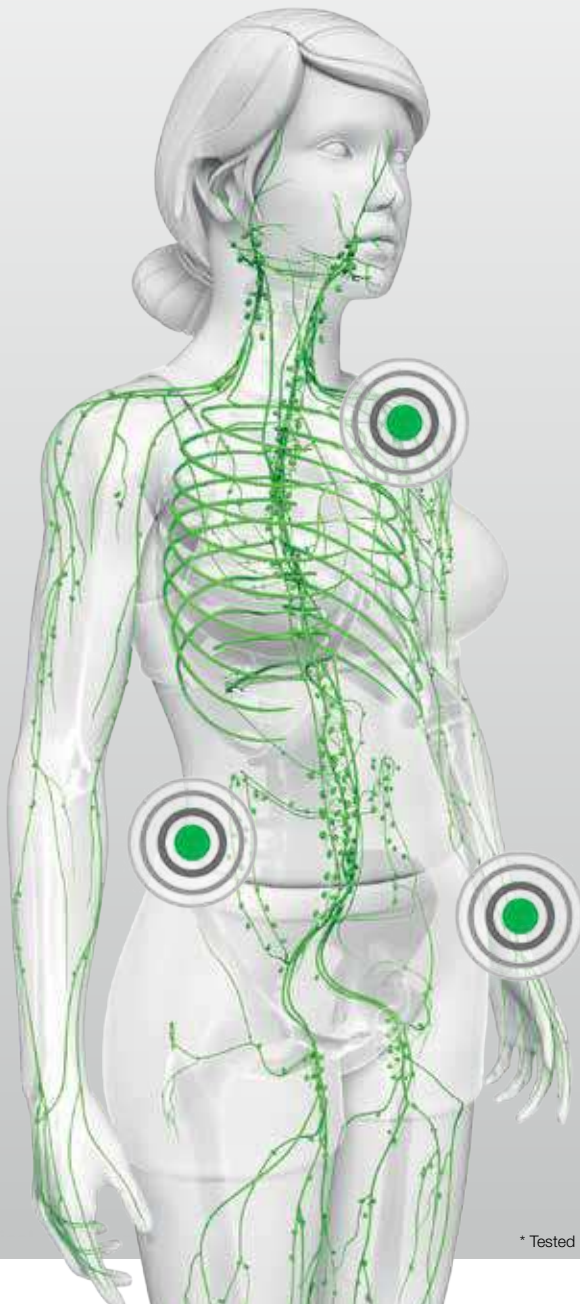


foam technology helps soften
tissue fibrosis and hardened skin



creates multipoint compression
profiles which help encourage
lymphatic reabsorption




versatile shapes and sizes
available off-the-shelf

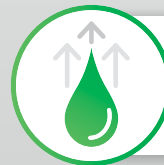

the special Coolcore fibre
helps wick away moisture



saves time from cutting
your own chip bags

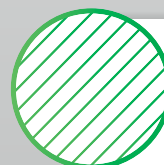


chevron channelling based on
manual lymphatic drainage (MLD)
principles of directing lymphatic
fluid to collateral pathways

* Tested by Hohenstein Institute, an accredited laboratory and research institute focused on testing and certifying textiles.

SwellSpots™

Head-Neck Pad, Versi-Face Pad, Curved Pad & Crescent Pad

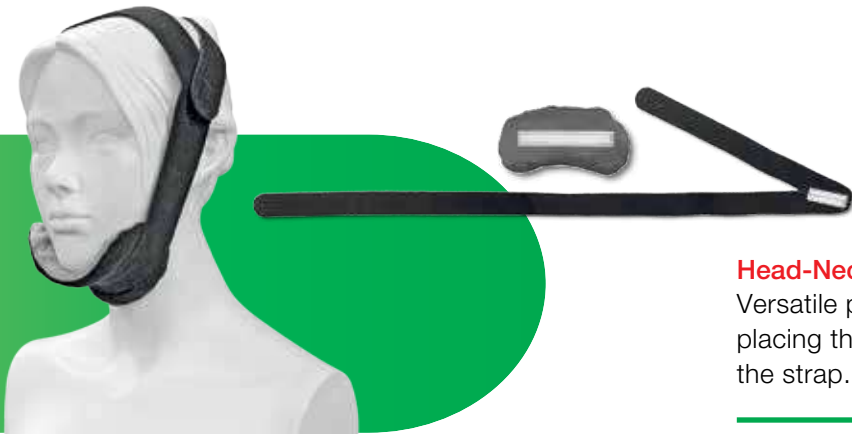

Head-Neck Pad (previously Reynolds)


Versatile pad designed for use on the face and neck by placing the Head-Neck Pad in various locations along the strap.

Pad width (cm)	Pad height (cm)	REF	Shipping unit
16.5	7.5	104 544	1/5

Versi-Face Pad

Versatile pad designed for use around the eyes and under the chin.

Width (cm)	Height (cm)	REF	Shipping unit
29	16	104 542	1/5

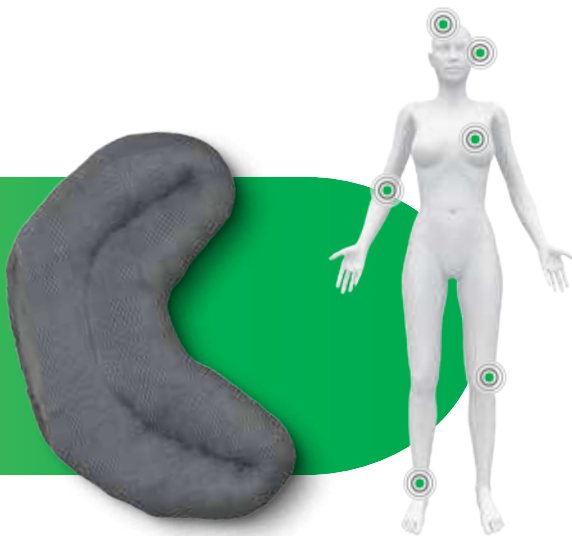

Curved Pad


Versatile pad designed for use around the ankle, patella, elbow, eyes and behind the ear.

Width (cm)	Height (cm)	REF	Shipping unit
5	13	104 527	1/5

Crescent Pad

Wedge-shaped pad designed to support lobules and skin folds.

Length (cm)	Height (cm)	Diameter	REF	Shipping unit
51	6.5	9	104 547	1/5
61	6.5	11	104 546	1/5
71	6.5	12	104 545	1/5

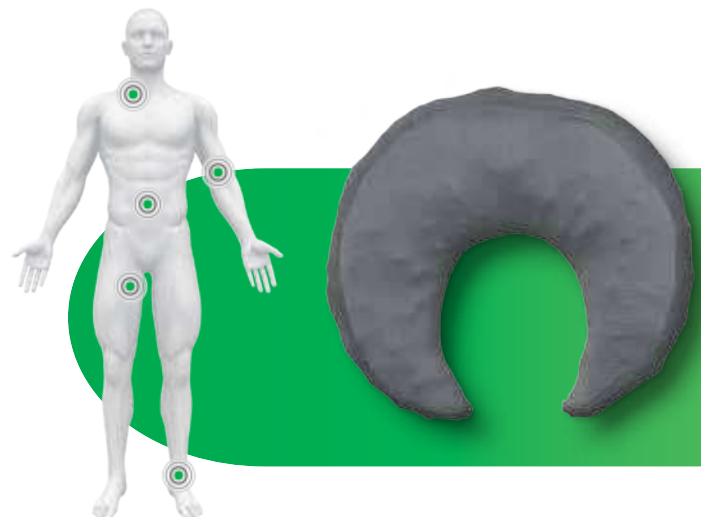

SwellSpots™

Carpal Hand Pad, Dorsal Pad, Rectangular-Tapered Pad & Knee-Elbow Shaper Pad

Carpal Hand Pad (previously Jo-Ju)

Pad designed to use at the thenar web space.

Width (cm)	Height (cm)	REF	Shipping unit
4.5	7	104 537	1/5



Dorsal Pad

Pad designed to use on the dorsum of either the hand or foot.

Width (cm)	Height (cm)	REF	Shipping unit
10	14	104 529	1/5
10	19	104 528	1/5



Rectangular-Tapered Pad (previously Katina Small & Large)


Tapered pad designed for use at the ulnar aspect of the forearm, the IT band of the lateral thigh, and the anterior surface of the calf.

Width (cm)	Height (cm)	REF	Shipping unit
11	36	104 534	1/5
18	42	104 533	1/5

Knee-Elbow Shaper Pad (previously LT Knee Shaper)


Versatile pad designed for use on the knee and elbow.

Girth range (cm)	REF	Shipping unit
35-65	104 557	1/5

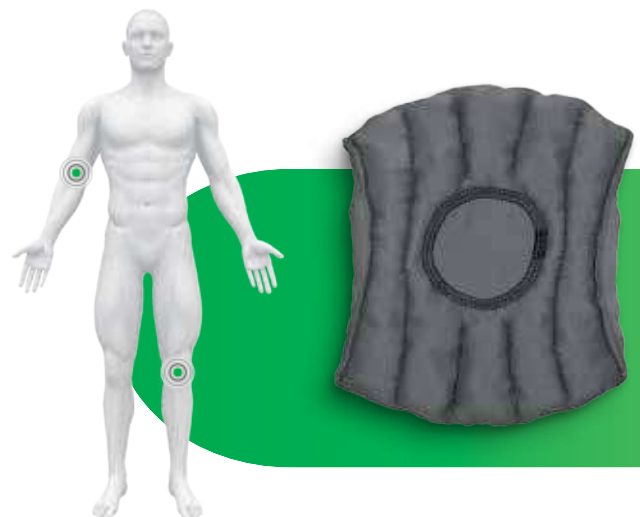

SwellSpots™

Oval Pad, Ring Pad, Kidney Pad & Tic-Tac Pad

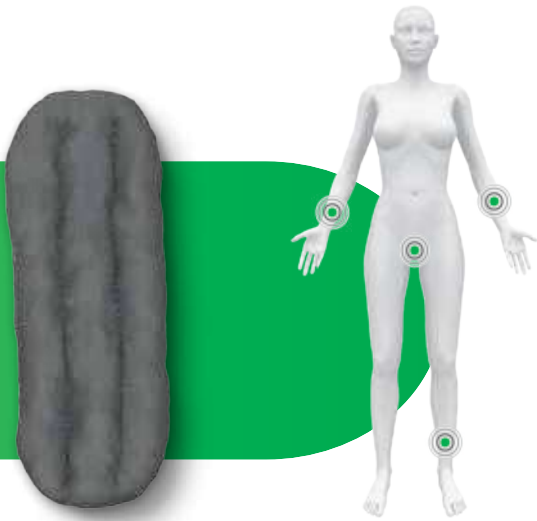

Oval Pad

Versatile pad designed for use inside the hand splints, along the ulnar aspect of the forearm, and the groin.

Width (cm)	Height (cm)	REF	Shipping unit
8	19	104 532	1/5

Ring Pad

Versatile pad designed to fill crevices or lift lobules at the ankle, wrist and abdominal pannuses. The Ring Pad is available with or without a VELCRO® brand hook closure.



Width (cm)	Height (cm)	Diameter	REF	Shipping unit
60	6.5	9	104 558*	1/5
60	6.5	9	104 559	1/5



Kidney Pad (previously Kimbe)

Versatile pad designed for use at the scapula, groin and neck.

Width (cm)	Height (cm)	REF	Shipping unit
14	24	104 538	1/5

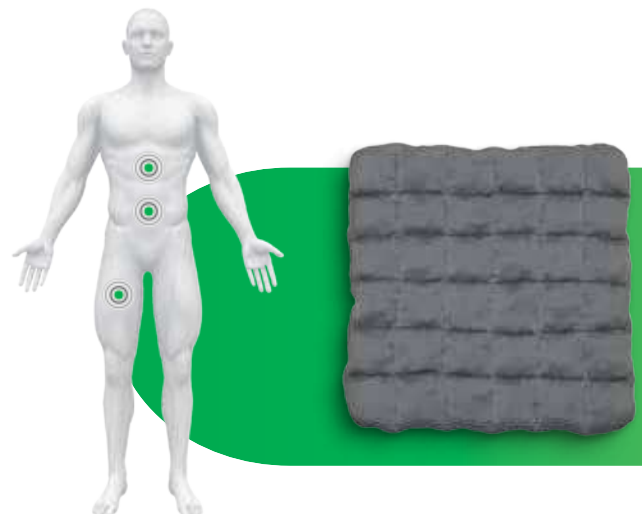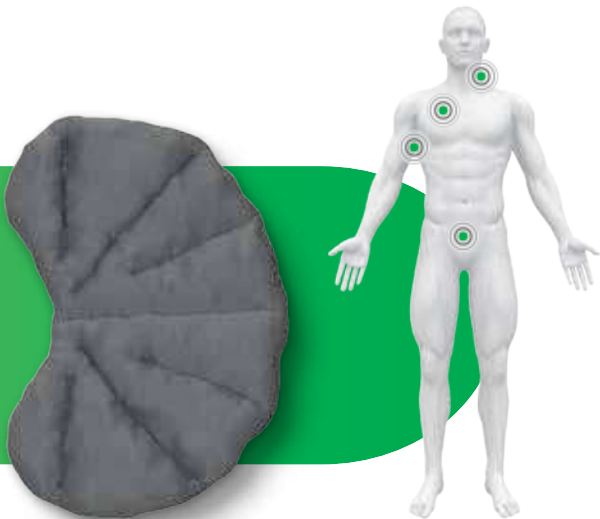

Tic-Tac Pad

Versatile pad designed to address larger areas of oedema and fibrosis, such as the back, abdomen and thigh.

Size (cm)	REF	Shipping unit
20 x 20	104 563	1/5
30 x 30	104 562	1/5
40 x 40	104 561	1/5

SwellSpots™

Labia Pad, Female Genital Pad, Scrotum Pad & Circular Pad



Labia Pad

Tapered pad designed for the female genital area that can be worn inside or over underwear.

Width (cm)	Height (cm)	REF	Shipping unit
33	53	104 551	1/5

Female Genital Pad (previously Sweet Spot)

Pad designed for use in the female genital area that can be placed inside or over underwear. The black side of the pad is made of non-skid material to help reduce shifting.

Width (cm)	Height (cm)	REF	Shipping unit
7	15	104 560	1/5



Scrotum Pad

Tapered pad designed for the male genital area that can be worn inside or over underwear.

Width (cm)	Height (cm)	REF	Shipping unit
34	53	104 552	1/5

Circular Pad (previously Sobol Small & Large)

Versatile pad designed for use on the chest wall, hip, anterior knee, posterior knee and abdomen.

Diameter (cm)	REF	Shipping unit
18	104 536	1/5
22	104 535	1/5



SwellSpots™

Full Bra Pad, Lateral Bra Pad, Breast Pad & Palmer Pad



Full Bra Pad

Pad designed for use from the lateral chest wall toward the anterior midline of the chest, and can be worn inside a bra.

Cup size*	REF	Shipping unit
A-B cup	104 556	1/5
B-C cup	104 555	1/5
C-D cup	104 554	1/5

Lateral Bra Pad

Pad designed for use on the lateral chest wall, and can be worn inside a bra.

Width (cm)	Height (cm)	REF	Shipping unit
10	16	104 553	1/5



Breast Pad

Pad designed specifically for use in the breast area, and can be worn inside a bra. Its seamless design allows the Breast Pad to be used on either side of the chest.

Cup size*	REF	Shipping unit
A-B cup	104 541	1/5
C-D cup	104 540	1/5
D-DD cup	104 539	1/5

Palmar Pad

Versatile pad designed for use on palmar surface of the hand, dorsum, concave shape of palm and inside a bra.

Width (cm)	Height (cm)	REF	Shipping unit
7	8	104 531	1/5
8	9	104 530	1/5



SwellSpots™

Toe Pad & Thorax-Thigh Pad



Toe Pad (previously We'a Toe Small, Medium & Large)


Unique digit spacers pad designed for comfort and separation of individual toes.

MTP circumference (cm)	REF	Shipping unit
<22	104 550	1/5
<25	104 549	1/5
<32	104 548	1/5

Thorax-Thigh Pad

Versatile pad designed for use at the chest, groin, medial thigh and shoulder.

Width (cm)	Height (cm)	REF	Shipping unit
52	22	104 543	1/5

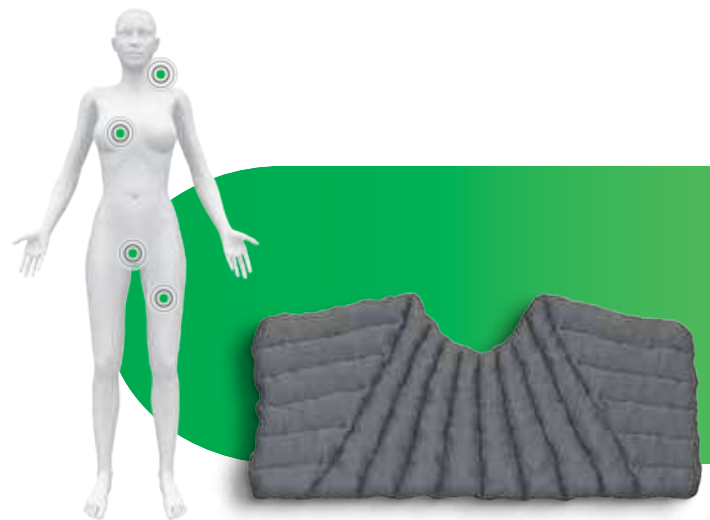

Care Instructions for SwellSpots

Washing

- machine-wash with mild detergent on permanent press
- do not use fabric softener, bleach or other additives
- do not dry-clean

Drying

- tumble-dry on permanent press
- do not iron

Materials

- blend of: polyurethane, nylon, polyester, elastane
- not made with natural rubber latex



Indications

- lymphoedema (when used with bandages, e.g. Rosidal K/other compression systems, e.g. ReadyWrap)
- oedema (when used with bandages, e.g. Rosidal K/other compression systems, e.g. ReadyWrap)

Precaution

- allergies or sensitivities to any of the product materials
- care should be taken in patients using anticoagulant medication to assess tolerance

Accessories

Accessories



Stocking Applicator

A plastic coated metal stocking applicator to assist in putting on compression or surgical stockings. The frame has foam covered handles. The frame will hold the stocking/sock sufficiently open to allow the foot to be inserted. The stocking still needs to be fed over the frame using the hands but may eliminate prolonged bending to reach the feet. Available in one size and colour.

Product Code: DV22400



VENOSAN® Silicone Dotted Donning Gloves

VENOSAN® silicone dotted donning gloves assist with applying stockings. Helps protect stockings from tears by finger nails. Available in sizes S/M and L/XL.


Product Codes: VC003 S/M
VC004 L/XL



VENOSAN® Coltex Roll-On Body Adhesive

The perfect way to hold up your hosiery - 24 hours a day. Use Coltex Adhesive, and you can be absolutely certain your stockings stay up at all times. Whatever the shape of your legs and whatever compression stockings you want to keep in place, this gentle but effective glue is a real confidence booster. Easy to apply and machine washable, Coltex Adhesive also contains hamamelis extracts to gently soothe your skin.

Product Code: KC67000



Accessories



Ezy-As Compression Stocking Applicator

A rigid plastic structure designed to assist with fitting closed and open pressure garments to upper and lower limbs. It is available in three sizes. Available in three sizes, with an optional handle attachment.

Product Codes: YELLOW SUT - SM 260 MED
 BLUE SUT - SM 261 LG
 RED SUT - SM 259 SM

VENOSAN® Socks-Jet Donning and Doffing Aid

The patented donning and doffing aid for medical compression stockings. Features easy to use doffing and donning bars for stocking application. Simple for use by nursing staff, also for bedridden patients.

Product Code: AV002



Eureka-On Compression Stocking Applicator

A compression stocking applicator for the leg or arm. It consists of a triangular shaped durable paper like slippery material with a central extended pull tab and eyelet holes on the sides for threading a thin tubular plastic lacing pin. Can also be used with pantyhose. Requires the use of two hands and for user to be able to reach their feet.

Product Code: EUREKA - ON



Accessories

Mediven Butler Off Compression Stocking Remover

This long handled device assists with the removal of compression stockings, socks and normal pantyhose. It is lightweight and is gentle on the skin as it has no sharp edges. It features a T-shaped thick grip handle, metal shaft and a curved tongue-like base with a hook. It can also double as a shoehorn.

Product Code: 900-A



Doff N' Donner

Designed to assist with the application of open and closed toe compression stockings. Suitable for fitting and removing garments on the upper and lower limbs below and above knee. Features two components: a 'doughnut' shaped soft silicon device, filled with a leak proof fluid and a cone with suction cup base that allows the compression garment to be rolled onto the 'doughnut' device ready for use.

Product Codes: DDCONE
DDDEVICE



Easy-Slide Compression Arm/Leg Stocking Applicator

A compression stocking applicator for open toe or arm sleeve stockings. The easy slide is cone shaped and made of a slippery, paper like material with a pull loop at the tapered end. The Easy-Slide is pulled onto the arm or leg and the stocking is then pulled over it. The Easy-Slide is removed by pulling the loop.



Leg	Shoe Size	Code
Small	34-37	ES10-S
Medium	38-41	ES20-M
Large	42-45	ES30-L
X-Large	>46	ES40-XL

Arm	Length	Code
Medium	55.5cm	ES70-M
Large	62.5cm	ES75-L

Accessories

Sama® Compression Therapy



Sama® Frottee Tubular Padding

- 95% cotton, 3% polyamide, 2% elastane
- Soft, time-saving tubular padding
- Multi-elastic applies with no wrinkling, creases or folds
- Snug and accurate fit - contours to all shapes, sizes and lengths
- Re-usable, will endure up to 20 washes at 40 degrees celcius depending on exudation, sterilisable
- Cut to desired length, no fraying
- Available in 10 metre rolls in four sizes (S, M, L, XL) and in four colours (white, pink, blue, green)

Functional benefits:

- Easy to apply, time-saving regime
- Increased comfort from padding layer, will not crease or wrinkle
- Allows the skin to breathe
- All products are washable and re-usable
- Cut and only use what you need from padding roll

Code	Size	Colour	Quantity
DV88103	Small	White	10m roll
DV88104	Medium	White	10m roll
DV88105	Large	White	10m roll
DV88106	X-Large	White	10m roll
DV88113	Small	Pink	10m roll
DV88114	Medium	Pink	10m roll
DV88115	Large	Pink	10m roll
DV88123	Small	Blue	10m roll
DV88124	Medium	Blue	10m roll
DV88125	Large	Blue	10m roll
DV88133	Small	Green	10m roll
DV88134	Medium	Green	10m roll
DV88135	Large	Green	10m roll

Size Chart

Small	15 - 35 cm circumference
Medium	30 - 55 cm circumference
Large	40 - 75 cm circumference
X-Large	50 - 95 cm circumference (White only)

© 2020

All information stated in this catalogue is correct at time of printing and subject to change without notice.

For the latest product information please visit www.venosan.com.au or call the sales team at Medical Accessories of Australia on 1300 527 127.





Unit 11/43 Lang Parade, Milton Queensland 4064 Australia
P. 1300 527 127 F. 07 3870 5944 E. sales@medicalaccessories.com.au
www.medicalaccessories.com.au



Australian distributors for

